



MISSOURI DEPARTMENT OF
**HEALTH &
SENIOR SERVICES**

Dexcom G6 Continuous Glucose Monitor Simulator:

- Explore Scenarios, Setup and Simulation tabs
 - Scenarios tab:
 - Select each scenario to view the Dexcom screen for that situation.
 - Select Scenarios button on top left of screen to return to the rest of the scenarios.
 - Setup tab:
 - Watch G6 Overview video (1:22)
 - Watch G6 treatment Decisions video (1:52)
 - G6 Insertion video (optional) (2:41)
 - Please note: The MO Nurse Practice Act allows for nurses to perform this task in accordance with Dr. orders and appropriate training.
 - Simulation tab
 - Click explore now and review video (1:26) – there is no sound with this one. 😊
 - After watching the video, please select “back” to return to the main screen for the next user.

Remember: When you have a student with any type of assistive device, always check the manufacturer’s website for more information on use of the device.

**Dexcom G6
Simulator**



**Diabetic Medical
Management Plan
(DMMP)**



**ADA Safe at School:
Guidelines for
the use of CGMs**




**ADA Safe at School:
Dosing from CGM Data**





MISSOURI DEPARTMENT OF
**HEALTH &
SENIOR SERVICES**

Medtronic MiniMed 770G Insulin Pump Simulator:

- You can use your mouse to click on the buttons of the insulin pump.
- Select the language by using the arrow buttons and clicking the white circle.
- Select time format
- Set time and date
- On “Congratulations! Start Up Wizard Complete” screen, press white circle to select “OK”
- Press white button and follow directions
- Enter a blood sugar of 120.
- Enter a bolus of 0.5U
- Can you find the preset basal rate?
- Bonus question: What is the max bolus this pump will allow as it is currently set?
- When finished with this simulator, please hit the refresh button on the top of the page. 

Omnipod DASH Insulin Pump Simulator:

- Select your language from the dropdown box.
- Click Continue
- Choose “Everyday Use”
- Enter a blood glucose – Don’t forget to choose a tag!
- Enter a number of carbs
- Give calculated bolus.
- Extended bolus is only used when a doctor orders it in the DMMP.
- When finished with this simulator, click on the 3 lines on top left of the simulator screen. Scroll to the bottom of the screen and select “Reset Application”.

**Medtronic MiniMed
770G Simulator**



**Medtronic Information
for School Nurses**



**School Nurse Pump
Training Skills Checklist**



**Omnipod DASH
Simulator**



**Tandem X2 Insulin
Pump: Information
for School Nurses**