

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

RECEIVED
By Carol Day at 12:48 pm, Jul 03, 2014

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 046408	PRINTER SN 08C.3556.310	DATE OF INSPECTION 07/02/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 1111 N. Glenstone Springfield		TIME OF INSPECTION 6:53 pm

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

- DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)
- TEMPERATURE OF ALCO SENSOR (10°C - 40°C)
- PRINTER WORKING PROPERLY
- TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

- SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER Guth Labs LOT # 13010 EXP. DATE 01/09/2015

SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34.0 SIMULATOR SN SD 2587 SIMULATOR EXP DATE 06/06/2015

- CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)
- 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
- 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
- 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 .102 TEST 2 .101 TEST 3 .102

- RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:
(DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	0	(0-.04)	0	(.05-.09)	0	(.10-.14)	0	(.15-.19)	0	(OVER .19)	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	------------	---

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

INSPECTING OFFICER

SIGNATURE <i>Mark Hartman</i> 160	PRINT NAME Mark Hartman
TYPE II PERMIT NUMBER/EXPIRATION DATE 220298 09/25/2014	TELEPHONE NUMBER (417) 575-8911

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office
2875 James Boulevard
Poplar Bluff, MO 63901



GUTH LABORATORIES, INC.

890 NORTH 67th STREET • HARRISBURG, PA 17111-4611 • TELEPHONE: 717-664-6470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 13010 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on January 14, 2013, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1218% (w/vol) ethyl alcohol. The expiration date for this lot number is January 9, 2015 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN122211-02 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

AS IV Serial no: 046408
Version no: 004C

TEST RECORD 00560

Temp Date Time 210L

Blank: 07/02/14 16:15 .000
Calibration Check: 07/02/14 16:15 .102

Subject Name

Subject I.D.

Operator Name, I.D.
M/ha ✓/60

Location

AS IV Serial no: 046408
Version no: 004C

TEST RECORD 00561

Temp Date Time 210L

Air Blank: 07/02/14 16:17 .000
Calibration Check: 07/02/14 16:17 .101

Subject Name

Subject I.D.

Operator Name, I.D.
M/ha ✓/60

Location

AS IV Serial no: 046408
Version no: 004C

TEST RECORD 00562

Temp Date Time 210L

Air Blank: 07/02/14 16:18 .000
Calibration Check: 07/02/14 16:18 .102

Subject Name

Subject I.D.

Operator Name, I.D.
M/ha ✓/60

Location

AS IV Serial no: 046408
Version no: 004C

TEST RECORD 00563

Temp Date Time 210L

Void: RFI
12 07/02/14 16:19

Subject Name

Subject I.D.

Operator Name, I.D.
M/ha ✓/60

Location

State of Missouri
DEPARTMENT OF HEALTH



P E R M I T
TYPE II



MARK HARTMAN

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV W/PRINTER

for the determination of the alcoholic content of blood from a sample of expired (alveolar) air. Issued under the provisions of sections 577.020 through 577.041, RSMo 1986.

Date 09/25/2012

Number 220298

Expires 09/25/2014

Director of State Public Health Laboratory

Director, Department of Health