



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

received 8/11/14-^{REPORT #7}cd

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and Send copy to Department of Health and Senior Services; retain original in department file. **REVIEWED** instrument is repaired. By Carol Day at 9:32 am, Aug 28, 2014

ALCO SENSOR IV SN 089178	PRINTER SN N/A	DATE OF INSPECTION 08/02/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 100 Washington Avenue, St. Louis MO 63102		TIME OF INSPECTION 1:30 pm

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)

TEMPERATURE OF ALCO SENSOR (10°C - 40°C)

PRINTER WORKING PROPERLY

TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER Guth Labs Inc. LOT # 14110 EXP. DATE 05/01/2016

SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34 SIMULATOR SN SD2735 SIMULATOR EXP DATE 01/15/2015

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
 Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE

0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE

0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

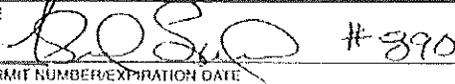
TEST 1 ← .098	TEST 2 ← .096	TEST 3 ← .097
---------------	---------------	---------------

RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	0	(.0-.04)	1	(.05-.09)	0	(.10-.14)	3	(.15-.19)	1	(OVER .19)	0
----------	---	----------	---	-----------	---	-----------	---	-----------	---	------------	---

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

INSPECTING OFFICER	
SIGNATURE  #890	PRINT NAME Thad Lucas
TYPE II PERMIT NUMBER/EXPIRATION DATE 240288 06/19/2016	TELEPHONE NUMBER (314) 655-1701

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office
 2875 James Boulevard
 Poplar Bluff, MO 63901



GUTH LABORATORIES, INC.

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-6470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 14110 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on May 5, 2014, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1206% (w/vol) ethyl alcohol. The expiration date for this lot number is May 1, 2016 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN122211-02 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.