



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

RECEIVED
 By Carol Day at 9:49 am, Feb 10, 2014

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 105452	PRINTER SN 097.3584.331	DATE OF INSPECTION 02/07/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 1200 Main Grandview		TIME OF INSPECTION 6:00 pm

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)

TEMPERATURE OF ALCO SENSOR (10°C - 40°C)

PRINTER WORKING PROPERLY

TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER Guth Lab LOT # 13100 EXP. DATE 04/23/2015

SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34.0 SIMULATOR SN DR3585 SIMULATOR EXP DATE 08/07/2014

CALIBRATION CHECK -- (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

- 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 → .101

TEST 2 → .100

TEST 3 → .100

RFI DETECTOR OPERATING

**INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:
 (DO NOT INCLUDE SELF-ADMINISTERED TESTS)**

REFUSALS	0	(0-.04)	0	(.05-.09)	0	(.10-.14)	0	(.15-.19)	0	(OVER .19)	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	------------	---

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

INSPECTING OFFICER

SIGNATURE
 180

PRINT NAME
 Brandon Grantham

TYPE II PERMIT NUMBER/EXPIRATION DATE
 230221 10/17/2015

TELEPHONE NUMBER
 (816) 316-4980

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office
 2875 James Boulevard
 Poplar Bluff, MO 63901

AS IV Serial no: 105452
Version no: 532B

TEST RECORD 00137 s/
Temp Date Time 210L

Air Blank: 02/07/14 18:26 .000
Calibration Check: 22 02/07/14 18:26 .101

Subject Name

Test 1

Subject I.D.

Operator Name, I.D.

Ganham 230221
Location

AS IV Serial no: 105452
Version no: 532B

TEST RECORD 00138 s/
Temp Date Time 210L

Air Blank: 02/07/14 18:28 .000
Calibration Check: 23 02/07/14 18:28 .100

Subject Name

Test 2

Subject I.D.

Operator Name, I.D.

Ganham 230221
Location

AS IV Serial no: 105452
Version no: 532B

TEST RECORD 00139 s/
Temp Date Time 210L

Air Blank: 02/07/14 18:29 .000
Calibration Check: 23 02/07/14 18:29 .100

Subject Name

Test 3

Subject I.D.

Operator Name, I.D.

Ganham 230221
Location

AS IV Serial no: 105452
Version no: 532B

TEST RECORD 00140 s/
Temp Date Time 210L

VOID: RFI
12 02/07/14 18:31

Subject Name

RFI Test

Subject I.D.

Operator Name, I.D.

Ganham 230221
Location



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM
INTOX DMT MAINTENANCE REPORT

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).
 Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.
 Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN 500014	NAME OF AGENCY Grandview Police Department	DATE OF INSPECTION 02/07/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 1200 Main St, Grandview, MO 64030		TIME OF INSPECTION 18:01:52

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
DATE AND TIME <u>02/07/2014 18:01:54</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.8°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>43.5°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS

<input checked="" type="checkbox"/> SIMULATOR STANDARD	<input type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE
--	---

<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>GUTH</u>	LOT # <u>13100</u>	EXP. DATE <u>04/23/2015</u>
---	--------------------	-----------------------------

<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) <u>34.0</u>	SIMULATOR SN <u>DR3585</u>	SIMULATOR EXP DATE <u>08/17/2014</u>
---	----------------------------	--------------------------------------

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
 Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.

<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.098	TEST 2: 0.099	TEST 3: 0.098
----------------------	----------------------	----------------------

PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 2	0-.04: 1	.05-.09: 0	.10-.14: 0	.15-.19: 0	OVER .19: 2
--------------------	-----------------	-------------------	-------------------	-------------------	--------------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

INSPECTING OFFICER

SIGNATURE 	PRINT FULL NAME BRANDON P GRANTHAM
---------------	--

TYPE II PERMIT NUMBER 230221	EXPIRATION DATE 10/17/2015	TELEPHONE NUMBER 816-316-4980
--	--------------------------------------	---

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services
 Southeast District Office
 2875 James Blvd, Poplar Bluff, MO 63901



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM
INTOX DMT MAINTENANCE REPORT

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).
 Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.
 Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN 500013	NAME OF AGENCY Grandview Police Department	DATE OF INSPECTION 02/07/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 1200 Main Street Grandview, MO 64030		TIME OF INSPECTION 17:55:56

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

DIAGNOSTIC RECORD

DATE AND TIME <u>02/07/2014 17:55:58</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.8°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>45.1°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS

SIMULATOR STANDARD COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER GUTH LOT # 13100 EXP. DATE 04/23/2015

SIMULATOR TEMP (34°C ± 0.2°C) 34.0 SIMULATOR SN DR3585 SIMULATOR EXP DATE 08/07/2014

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
 Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.

0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE

0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE

0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.096 TEST 2: 0.096 TEST 3: 0.097

PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 7 0-.04: 2 .05-.09: 0 .10-.14: 1 .15-.19: 2 OVER .19: 2

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

INSPECTING OFFICER

SIGNATURE _____ PRINT FULL NAME **BRANDON P GRANTHAM**

TYPE II PERMIT NUMBER 180 EXPIRATION DATE **10/17/2015** TELEPHONE NUMBER **816-316-4980**

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services
 Southeast District Office
 2875 James Blvd, Poplar Bluff, MO 63901



GUTH LABORATORIES, INC.

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 13100 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on April 29, 2013, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1214% (w/vol) ethyl alcohol. The expiration date for this lot number is April 23, 2015 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN122211-02 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.