

RECEIVED
By Carol Day at 1:49 pm, Jul 14, 2014



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY
BREATH ALCOHOL PROGRAM
INTOX EC/IR II MAINTENANCE REPORT

REPORT #3

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12818	NAME OF AGENCY ST. ANN POLICE DEPT.	DATE OF INSPECTION 07/06/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 10405 Mo-180 St Ann		TIME OF INSPECTION 23:53 CDT

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

DIAGNOSTIC RECORD

<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST
<input checked="" type="checkbox"/> ETH CHECK	

BREATH ANALYZER ACCURACY STANDARDS

<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE
<input checked="" type="checkbox"/> STANDARD SUPPLIER intox	LOT# ag410601 EXP. DATE 04/16/2016
<input type="checkbox"/> SIMULATOR TEMP (34°C +0.2°C)	SIMULATOR S/N SIMULATOR EXP DATE

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within +5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1 = 0.100 g/210L	TEST 2 = 0.100 g/210L	TEST 3 = 0.099 g/210L
-----------------------	-----------------------	-----------------------

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS	0	0-.04	1	.05-.09	0	.10-.14	0	.15-.19	0	OVER .19	0
----------	---	-------	---	---------	---	---------	---	---------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

Instrument tested with DOH specifications

INSPECTING OFFICER

SIGNATURE ▶ <i>William Goodrich</i>	PRINT FULL NAME GOODRICH, WILLIAM
TYPE II PERMIT NUMBER 240245	EXPIRATION DATE 05/19/2016
	TELEPHONE NUMBER (314) 427-8000

RETURN COMPLETED REPORT TO THE:
Breath Alcohol Program, Missouri Department of Health and Senior Services,
Southeast District Office, 2875 James Blvd, Poplar Bluff, MO 63901



GUTH LABORATORIES, INC.

690 NORTH 67th STREET • HARRISBURG, PA 17111-4611 • TELEPHONE: 717-664-6470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 13280 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on October 18, 2013, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1217% (w/vol) ethyl alcohol. The expiration date for this lot number is October 16, 2015 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN122211-02 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.