



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM

RECEIVED
 By Carol Day at 2:35 pm, Dec 03, 2014

INTOX EC/IR II MAINTENANCE REPORT

REPORT #3

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12826	NAME OF AGENCY Franklin Co Sheriffs Off	DATE OF INSPECTION 11/24/2014
LOCATION OF INSTRUMENT (STREET AND CITY) #1 Bruns Lane Union, MO		TIME OF INSPECTION 17:29 CST
CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.		
<input checked="" type="checkbox"/> DIAGNOSTIC RECORD		
<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK	
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK	
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK	
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK	
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK	
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST	
<input checked="" type="checkbox"/> ETH CHECK		
BREATH ANALYZER ACCURACY STANDARDS		
<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE	
<input checked="" type="checkbox"/> STANDARD SUPPLIER	INTOXIMETERS	LOT# AG423201 EXP. DATE 08/20/2016
<input type="checkbox"/> SIMULATOR TEMP (34°C ±0.2°C)	SIMULATOR S/N	SIMULATOR EXP DATE
<input checked="" type="checkbox"/> CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)		
Run three tests using a standard solution. All three tests must be within +5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)		
<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE		
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE		
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE		
TEST 1 → 0.100 g/210L	TEST 2 → 0.100 g/210L	TEST 3 → 0.100 g/210L
INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:		
REFUSALS 0	0-.04 0	.05-.09 0
.10-.14 0	.15-.19 0	OVER .19 0
LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).		
Second check, wrong card used.		
INSPECTING OFFICER		
SIGNATURE 		PRINT FULL NAME RICHARDSON, MICHAEL
TYPE II PERMIT NUMBER 240127	EXPIRATION DATE 04/03/2016	TELEPHONE NUMBER (636) 583-2560
RETURN COMPLETED REPORT TO THE:		
Breath Alcohol Program, Missouri Department of Health and Senior Services, Southeast District Office, 2875 James Blvd, Poplar Bluff, MO 63901		



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM

INTOX EC/IR II MAINTENANCE REPORT

REPORT #3

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12826	NAME OF AGENCY Franklin Co Sheriffs Off	DATE OF INSPECTION 11/24/2014
----------------------------	--	----------------------------------

LOCATION OF INSTRUMENT (STREET AND CITY) #1 Bruns Lane Union, MO	TIME OF INSPECTION 17:08 CST
---	---------------------------------

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST
<input checked="" type="checkbox"/> ETH CHECK	

BREATH ANALYZER ACCURACY STANDARDS	
<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE

<input checked="" type="checkbox"/> STANDARD SUPPLIER	INTOXIMETERS	LOT# AG423201	EXP. DATE 08/20/2016
---	--------------	---------------	----------------------

<input type="checkbox"/> SIMULATOR TEMP (34°C ±0.2°C)	SIMULATOR S/N	SIMULATOR EXP DATE
---	---------------	--------------------

<input checked="" type="checkbox"/> CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)	
Run three tests using a standard solution. All three tests must be within +5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)	
<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE	
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE	
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE	

TEST 1 0.101 g/210L	TEST 2 0.101 g/210L	TEST 3 0.100 g/210L
---------------------	---------------------	---------------------

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:											
REFUSALS	0	0-.04	30	.05-.09	0	.10-.14	2	.15-.19	1	OVER .19	0

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).	

INSPECTING OFFICER	
SIGNATURE 	PRINT FULL NAME RICHARDSON, MICHAEL
TYPE II PERMIT NUMBER 230343	TELEPHONE NUMBER (636) 583-2560
EXPIRATION DATE 12/31/2015	

RETURN COMPLETED REPORT TO THE:
 Breath Alcohol Program, Missouri Department of Health and Senior Services,
 Southeast District Office, 2875 James Blvd, Poplar Bluff, MO 63901



Airgas USA LLC (LAB)
 3500 Bernard Street
 St. Louis, Mo. 63103
 Ph: (314) 533-3100
 Fax: (314) 533-7328

Certificate of Analysis

Customer Name
 Intoximeters, Inc.
 2081 Craig Road
 St. Louis, Mo 63146

Test Date: 20-Aug-2014

Lot # AG423201

<u>Exp. Date</u> 20-Aug-2016	<u>Cyl. Type</u> 108	<u>Component</u> Ethanol Nitrogen	<u>Certified Concentration</u> 0.100 ± 2% BrAC (272 ppm) Balance
--	--------------------------------	--	---

Certification Traceable to N.I.S.T. RGM Ethanol Standards:


<u>Serial No.</u>	<u>Concentration</u>	<u>Serial No.</u>	<u>Concentration</u>
EB0010581	391.8 ppm	EB0010603	392.5 ppm
EB0010570	259.8 ppm	EB0010559	258.9 ppm
EB0010285	209.0 ppm	EB0010595	208.9 ppm
EB0010561	103.7 ppm	EB0010562	104.9 ppm
EB0010681	52.22 ppm	EB0010579	52.94 ppm

Analytical Method: NDIR

Digitally signed by Quality Control
 Date: 2014.08.22 12:04:20 -05:00
 Reason: Dry gas standard certification of analysis
 Location: Airgas USA LLC (Lab)

Analyst: 
 Rod Marsala

ISO 17025:2005 A2LA accredited. Certificate Number 2989.01

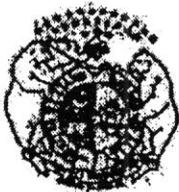

STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

2

**PERMIT
 TYPE II**

MICHAEL RICHARDSON

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER, INTOX EC/IR II

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 806.111 through 806.119 RSMo.

DATE 4/3/2014

NUMBER 240127

EXPIRES 4/3/2016

W. W. ...

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

Gail Vesterly

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

(48-116-10)