

MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 STATE PUBLIC HEALTH LABORATORY  
 BREATH ALCOHOL PROGRAM  
**DATAMASTER MAINTENANCE REPORT**

**RECEIVED**  
 By Carol Day at 9:46 am, Oct 15, 2014

REPORT #6

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).  
 Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.  
 Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

DATAMASTER SN 201191	NAME OF AGENCY Claycomo Police Department	DATE OF INSPECTION 10/14/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 115 E. 69 Hwy. Claycomo		TIME OF INSPECTION 11:32 am

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC CHECK (PRINTOUT ATTACHED)	DATE AND TIME (from printout) <u>10/14/14 11:32</u>
<input checked="" type="checkbox"/> COMPUTER	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTERS
<input checked="" type="checkbox"/> HEATERS SAMPLE CHAMBER <u>49°C</u>	<input checked="" type="checkbox"/> QUARTZ STANDARD
<input checked="" type="checkbox"/> FLOW DETECTOR	<input checked="" type="checkbox"/> CALIBRATION
<input checked="" type="checkbox"/> PUMP HIGH SPEED	<input checked="" type="checkbox"/> PRINTER

INDICATOR LIGHTS

SIMULATOR SOLUTION SUPPLIER RepCo Marketing LOT # 13002 EXP. DATE 06/19/2015

SIMULATOR TEMP (34°C ± 0.2°C) 34.0 °C SIMULATOR SN SD1353 EXP. DATE 02/27/2014

CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

- 0.100% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
- 0.080% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
- 0.040% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1 <u>.100</u>	TEST 2 <u>.098</u>	TEST 3 <u>.100</u>
--------------------	--------------------	--------------------

PERFORM R.F.I. TEST (PRINTOUT ATTACHED)

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:  
 (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	0	(0-.04)	0	(.05-.09)	0	(.10-.14)	2	(.15-.19)	0	OVER .19	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).  
 Operates within D.H.S.S. standards.

**INSPECTING OFFICER**

SIGNATURE 	PRINT FULL NAME Matthew E. Beets
---------------	-------------------------------------

TYPE II PERMIT NUMBER/EXPIRATION DATE 220401 12/20/2014	TELEPHONE NUMBER (816) 452-4613
--	------------------------------------

RETURN COMPLETED REPORT TO THE: **Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office  
 2875 James Blvd.  
 Poplar Bluff, MO 63901**

## CERTIFICATE OF ANALYSIS

**MANUFACTURER AND SUPPLIER:** RepCo Marketing, Inc.

**LOT NUMBER:** 13002

**EXPIRATION DATE:** June 19, 2015 at 11:59 p.m.

RepCo Marketing, Inc. certifies the following:

RepCo Marketing, Inc. manufactured, tested and supplied Lot Number 13002 of Alcohol Certified Solution for simulators. Random samples of said lot number were analyzed by an independent laboratory utilizing a gas chromatograph and found to contain .1217 gms/dl +/- .003 gms/dl wt/vol ethanol (95% Confidence).

The alcohol and distilled water used in the solution were found to be free of any interfering substance.

This solution will produce a vapor alcohol value of .100 +/- 3% gms/210L Breath when heated to 34 Degrees Celsius +/- 0.2 Degrees Celsius in a simulator (95% Confidence).

The date of manufacture for this lot number is June 20, 2013  
The expiration date for this lot number is June 19, 2015 at  
11:59 p.m.

This document is a true representation of the original Certificate of Analysis.



Cecil B. Garner, President  
RepCo Marketing, Inc.

Face This Side Down - This Edge In First

**BAC DataMaster**  
**Evidence Ticket**

STATE OF MISSOURI  
CLAYCOMB POLICE DEPARTMENT

BAC DATAMASTER SERIAL NUMBER 201191  
10/14/14

ARREST TIME: 11:30  
SUBJECT NAME:  
RFI/TEST  
DOB: 01/01/91 SEX: M  
STATE/D.L.: MO/1294567  
ARRESTING OFFICER:  
DEETS/MATTHEW/E  
OFFICER I.D.: 119  
TESTING OFFICER:  
DEETS/MATTHEW/E  
OFFICER I.D.: 119  
PERMIT NUMBER: 220401  
EXPIRATION DATE: 12/28/14  
MISCELLANEOUS DATA:  
RFI TEST MONTHLY MAINT.

--- BREATH ANALYSIS ---

BLANK TEST	.000	11:47
INTERNAL STANDARD	VERIFIED	11:47
RFID INTERFERENCE		

Operator Signature



# BAC DataMaster Evidence Ticket

STATE OF MICHIGAN  
CLAYTON POLICE DEPARTMENT

BAC DATAMASTER SERIAL NUMBER 261191  
12/14/14

TESTING OFFICER:  
REETS/MATTINGALE  
OFFICER I.D.# 119  
PERMIT NUMBER: 222090  
EXPIRATION DATE: 12/28/14  
UNUSUAL COMMENTS:  
NONE

--- SUPERVISOR NODE ---

BLANK TEST	.000	11:30
INTERNAL STANDARD	VERIFIED	11:30
INTERNAL STANDARD	.100	11:32
BLANK TEST	.000	11:35
EXTERNAL STANDARD	.090	11:40
BLANK TEST	.000	11:40
EXTERNAL STANDARD	.100	11:41
BLANK TEST	.000	11:41

N = 3  
SIG. = .1  
AVG. = .0790

# BAC DataMaster Evidence Ticket

STATE OF MICHIGAN  
CLAYTON POLICE DEPARTMENT

BAC DATAMASTER SERIAL NUMBER 261191  
12/14/14  
11:32

--- DIAGNOSTIC CHECK ---

COMPUTER:	OKAY
PROGRAM (04-07-2009):	OKAY
HEATERS	
SAMPLE CHAMBER:	49c
FLOW DETECTOR:	OKAY
PUMP	
HIGH SPEED:	OKAY
DETECTOR:	OKAY
FILTERS:	OKAY
QUARTZ STANDARD:	OKAY
CALIBRATION:	OKAY

PRINTER TEST

!"#\$%&'()\*+,-./0123456789:;@<=>?@ABCDEFGHI  
JKLMNOPQRSTUVWXYZ[\]^\_`abcdefghijklmnopq  
rstuvwxyz{|}~"

Operator Signature



2208-02

Operator Signature



2208-02



PERMIT  
TYPE II



MATTHEW E BEETS

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field repairs, and operate the following breath analyzer(s):

DATAMASTER

for the determination of the alcoholic content of blood from a sample of expired (alveolar) air. Issued under the provisions of sections 577.020 through 577.041, RSMo 1986.

Date 12/20/2012

Number 220401

Expires 12/20/2014

Director of State Public Health Laboratory

ACTING DIRECTOR

Director, Department of Health



## Simulator Calibration Report

This calibration report is to certify the alcohol reference simulator listed below has been examined and tested using standards traceable to the National Institute of Standards and Technology (NIST) in accordance with the standards set by the Missouri Department of Health and Senior Services Rules and Regulations:  
19 CSR 25-30.051 (4).

Checked: 2/27/2014      Expires: 02/27/2015  
Digital Therm. SN:094948    Temp:33.98  
MSC Tech:DRL  
Agency: Claycomo Police Department  
SD1353



Technician Printed Name: Dan Lucas

Technician Signature: 

Date: 02/27/2014

Contact: Missouri Safety Center  
Breath-Alcohol Instrument Training Program  
660-543-4834