



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM
DATAMASTER MAINTENANCE REPORT

RECEIVED 2/8/14 REPORT #6

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).
 Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.
 Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

REVIEWED
 By Carol Day at 1:07 pm, Feb 24, 2014

| | | |
|--|---|----------------------------------|
| DATAMASTER SN 201288 | NAME OF AGENCY Campbell Police Dept. | DATE OF INSPECTION 02/02/2014 |
| LOCATION OF INSTRUMENT (STREET AND CITY) 204 W. Grand Cambell, MO 63933 | | TIME OF INSPECTION 7:06 am |

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

| | |
|--|---|
| <input checked="" type="checkbox"/> DIAGNOSTIC CHECK (PRINTOUT ATTACHED) | DATE AND TIME (from printout) 02/02/14 |
| <input checked="" type="checkbox"/> COMPUTER | <input checked="" type="checkbox"/> DETECTOR |
| <input checked="" type="checkbox"/> PROGRAM | <input checked="" type="checkbox"/> FILTERS |
| <input checked="" type="checkbox"/> HEATERS SAMPLE CHAMBER 49 °C | <input checked="" type="checkbox"/> QUARTZ STANDARD |
| <input checked="" type="checkbox"/> FLOW DETECTOR | <input checked="" type="checkbox"/> CALIBRATION |
| <input checked="" type="checkbox"/> PUMP HIGH SPEED | <input checked="" type="checkbox"/> PRINTER |

INDICATOR LIGHTS

SIMULATOR SOLUTION SUPPLIER Repro LOT # 13002 EXP. DATE 06/19/2015

SIMULATOR TEMP (34°C ± 0.2°C) 34 °C SIMULATOR SN SD1743 EXP. DATE 01/16/2015

CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

- 0.100% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
- 0.080% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
- 0.040% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

| | | |
|--|--|--|
| TEST 1 <input checked="" type="checkbox"/> .096% | TEST 2 <input checked="" type="checkbox"/> .095% | TEST 3 <input checked="" type="checkbox"/> .096% |
|--|--|--|

PERFORM R.F.I. TEST (PRINTOUT ATTACHED)

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:
 (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

| | | | | | | | | | | | |
|----------|---|---------|---|-----------|---|-----------|---|-----------|---|----------|---|
| REFUSALS | 0 | (0-.04) | 0 | (.05-.09) | 0 | (.10-.14) | 0 | (.15-.19) | 0 | OVER .19 | 0 |
|----------|---|---------|---|-----------|---|-----------|---|-----------|---|----------|---|

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

Instrument is within D.O.H. specifications. Simulator solution Repco lot# 13002 exp. date June 19th 2015. Vapor alcohol value .100+/- 3%.

INSPECTING OFFICER

| | |
|--|------------------------------------|
| SIGNATURE <i>Weldon Wallace</i> | PRINT FULL NAME Weldon Wallace |
| TYPE II PERMIT NUMBER/EXPIRATION DATE 220240 09/07/2014 | TELEPHONE NUMBER (573) 246-2511 |

RETURN COMPLETED REPORT TO THE: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office
 2875 James Blvd.
 Poplar Bluff, MO 63901

REPCO MARKETING INC.

3101-108 STONYBROOK DRIVE
RALEIGH, N.C. 27604
919-876-5400

CERTIFICATE OF ANALYSIS

MANUFACTURER AND SUPPLIER: RepCo Marketing, Inc.

LOT NUMBER: 13002

EXPIRATION DATE: June 19, 2015 at 11:59 p.m.

RepCo Marketing, Inc. certifies the following:

RepCo Marketing, Inc. manufactured, tested and supplied Lot Number 13002 of Alcohol Certified Solution for simulators. Random samples of said lot number were analyzed by an independent laboratory utilizing a gas chromatograph and found to contain .1217 gms/dl +/- .003 gms/dl wt/vol ethanol (95% Confidence).

The alcohol and distilled water used in the solution were found to be free of any interfering substance.

This solution will produce a vapor alcohol value of .100 +/- 3% gms/210L Breath when heated to 34 Degrees Celsius +/- 0.2 Degrees Celsius in a simulator (95% Confidence).

The date of manufacture for this lot number is June 20, 2013
The expiration date for this lot number is June 19, 2015 at 11:59 p.m.

This document is a true representation of the original Certificate of Analysis.

Cecil B. Garner

Cecil B. Garner, President
RepCo Marketing, Inc.

State of Missouri
DEPARTMENT OF HEALTH



P E R M I T
TYPE II



WELDON WALLACE

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field repairs, and operate the following breath analyzer(s):

DATAMASTER

for the determination of the alcoholic content of blood from a sample of expired (alveolar) air. Issued under the provisions of sections 577.020 through 577.041, RSMo 1986.

Date 09/07/2012

Number 220240

Expires 09/07/2014

Director of State Public Health Laboratory

Director, Department of Health

BAC DataMaster
Evidence Ticket

STATE OF MISSOURI
CAMPELL POLICE DEPARTMENT


BAC DATAMASTER SERIAL NUMBER 2012218
00/02/14

ARREST TIME: 07:00
 SUBJECT NAME:
 DOB: 10/10/76 SEX: M
 STATE: D.L.# MO-123456
 ARRESTING OFFICER:
 WALLACE/WELDON/L
 OFFICER I.D.# 23
 TESTING OFFICER:
 WALLACE/WELDON/L
 OFFICER I.D.# 23
 PENALTY NUMBER: 200540
 EXPIRATION DATE: 05/07/14
 MISCELLANEOUS DATA:
 MAINTENANCE CHECK

--- BREATH ANALYSIS ---

| | | |
|--------------------|----------|-------|
| BLIND TEST | .004 | 07:30 |
| INTERNAL STANDARD | VERIFIED | 07:33 |
| RADIO INTERFERENCE | | |

OPERATOR SIGNATURE

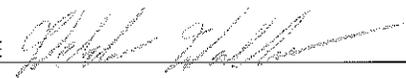

Card Stock No.
60021

REORDER ALL SUPPLIES FROM N.P.A.S.
P.O. BOX 1435, MANSFIELD, OH 44901