



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
STATE PUBLIC HEALTH LABORATORY  
BREATH ALCOHOL PROGRAM

**DATAMASTER MAINTENANCE REPORT**

RECEIVED 6/11/14 **REPORT #6**

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

**REVIEWED**  
By Carol Day at 1:54 pm, Jun 26, 2014

DATAMASTER SN 204143	NAME OF AGENCY MISSOURI STATE HIGHWAY PATROL	DATE OF INSPECTION 06/05/2014
LOCATION OF INSTRUMENT (STREET AND CITY) JEFFERSON COUNTY #34- NORTH ZONE OFFICE		TIME OF INSPECTION 1:07 pm

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC CHECK (PRINTOUT ATTACHED)	DATE AND TIME (from printout) 06/05/2014 1307 hrs
<input checked="" type="checkbox"/> COMPUTER	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTERS
<input checked="" type="checkbox"/> HEATERS SAMPLE CHAMBER 49°C	<input checked="" type="checkbox"/> QUARTZ STANDARD
<input checked="" type="checkbox"/> FLOW DETECTOR	<input checked="" type="checkbox"/> CALIBRATION
<input checked="" type="checkbox"/> PUMP HIGH SPEED	<input checked="" type="checkbox"/> PRINTER

<input checked="" type="checkbox"/> INDICATOR LIGHTS
<input checked="" type="checkbox"/> SIMULATOR SOLUTION SUPPLIER <u>GUTH LABORATORIES</u> LOT # <u>13290</u> EXP. DATE <u>10/29/2015</u>
<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) <u>33.96</u> °C SIMULATOR SN <u>1024</u> EXP. DATE <u>10/21/2014</u>

**CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)**

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

0.100% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE  
 0.080% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE  
 0.040% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1 <input checked="" type="checkbox"/> .096	TEST 2 <input checked="" type="checkbox"/> .096	TEST 3 <input checked="" type="checkbox"/> .097
---	---	---

**PERFORM R.F.I. TEST (PRINTOUT ATTACHED)**

**INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)**

REFUSALS	0	(.0-.04)	1	(.05-.09)	0	(.10-.14)	0	(.15-.19)	0	OVER .19	1
----------	---	----------	---	-----------	---	-----------	---	-----------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

THIS UNIT MEETS ALL MO DHSS SPECIFICATIONS.

GUTH LABORATORIES- LOT:13290 - BOTTLE: 610- EXPIRATION DATE:10/29/2015

**INSPECTING OFFICER**

SIGNATURE ▶	PRINT FULL NAME M. T. ANGRISANI
TYPE II PERMIT NUMBER/EXPIRATION DATE 220156 06/26/2014	TELEPHONE NUMBER (636) 300-2800

RETURN COMPLETED REPORT TO THE: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office  
2875 James Blvd.  
Poplar Bluff, MO 63901

**GUTH LABORATORIES, INC.**

880 NORTH 87th STREET • HARRISBURG, PA 17111-4811 • TELEPHONE: 717-664-6470

**CERTIFICATE OF ANALYSIS**

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 13290 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on October 31, 2013, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1202% (w/vol) ethyl alcohol. The expiration date for this lot number is October 29, 2015 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President  
GUTH LABORATORIES, INC.

***NIST Traceability:***

*Testing was conducted using Cerilliant Reference Standard lot number FN122211-02 whose values are traceable to NIST.*

*All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.*



State of Missouri  
DEPARTMENT OF HEALTH



PERMIT  
TYPE II



MATTHEW T ANGRISANI

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field repairs, and operate the following breath analyzer(s):

DATAMASTER

for the determination of the alcoholic content of blood from a sample of expired (alveolar) air, issued under the provisions of sections 577.020 through 577.041, RSMo 1996.

Date: 06/26/2012

Number: 220156

Expires: 06/26/2014

MO 826-9771 (2-08)

Director of State Public Health Laboratory

Director, Department of Health

Lsh. 4 (80-22)