



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 STATE PUBLIC HEALTH LABORATORY  
 BREATH ALCOHOL PROGRAM  
**DATAMASTER MAINTENANCE REPORT**

**RECEIVED**  
 By Carol Day at 8:06 am, Aug 14, 2014

REPORT #6

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).  
 Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.  
 Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

DATAMASTER SN 204159 (Inv# 127276)	NAME OF AGENCY Missouri State Highway Patrol	DATE OF INSPECTION 08/02/2014
LOCATION OF INSTRUMENT (STREET AND CITY) 415 W. 3rd Street Platte City, Missouri (Platte County Detention)		TIME OF INSPECTION 7:13 am

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC CHECK (PRINTOUT ATTACHED)	DATE AND TIME (from printout) <u>08-02-2014 0713</u>
<input checked="" type="checkbox"/> COMPUTER	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTERS
<input checked="" type="checkbox"/> HEATERS SAMPLE CHAMBER <u>50</u> °C	<input checked="" type="checkbox"/> QUARTZ STANDARD
<input checked="" type="checkbox"/> FLOW DETECTOR	<input checked="" type="checkbox"/> CALIBRATION
<input checked="" type="checkbox"/> PUMP HIGH SPEED	<input checked="" type="checkbox"/> PRINTER

<input checked="" type="checkbox"/> INDICATOR LIGHTS
<input checked="" type="checkbox"/> SIMULATOR SOLUTION SUPPLIER <u>RepCo Marketing</u> LOT # <u>13280</u> EXP. DATE <u>10/16/2015</u>

<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) <u>34</u> °C SIMULATOR SN <u>G11071</u> EXP. DATE <u>12/04/2014</u>
---

CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

0.100% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE  
 0.080% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE  
 0.040% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1  .099	TEST 2  .100	TEST 3  .100
--------------	--------------	--------------

PERFORM R.F.I. TEST (PRINTOUT ATTACHED)

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:  
 (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	0	(0-.04)	0	(.05-.09)	4	(.10-.14)	1	(.15-.19)	1	OVER .19	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

Instrument is operating within DHSS specifications.

<b>INSPECTING OFFICER</b>	
SIGNATURE 	PRINT FULL NAME Trooper T.I. Goolsby
TYPE II PERM. NUMBER/EXPIRATION DATE 240092 03/11/2016	TELEPHONE NUMBER (816) 622-0800

RETURN COMPLETED REPORT TO THE:  
 Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office  
 2875 James Blvd.  
 Poplar Bluff, MO 63901



## GUTH LABORATORIES, INC.

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

### CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number **13280** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **October 18, 2013**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.1217%** (w/vol) ethyl alcohol. The expiration date for this lot number is **October 16, 2015** at 11:59 PM.

When used in a calibrated Simulator, operating at  $34^{\circ}\text{C} \pm .2^{\circ}\text{C}$ , this solution will give a breath alcohol analysis instrument reading of **0.100 g/210L**  $\pm$  3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President  
GUTH LABORATORIES, INC.

***NIST Traceability:***

*Testing was conducted using Cerilliant Reference Standard lot number FN122211-02 whose values are traceable to NIST.*

*All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.*



**BAC DataMaster**  
Evidence Ticket

MISSOURI STATE HIGHWAY PATROL  
BAC INSTRUMENT SERIAL NUMBER 204159  
08/02/14

TESTING OFFICER:  
GONULSKY, T  
OFFICER J.M.A. 859  
PERMIT NUMBER 24092  
EXPIRATION DATE 02/11/16  
MISCELLANEOUS DATA  
J.M.

TELEVISION NOTE

BLANK TEST	VERIFIED	07:21
INTERNAL STANDARD	.000	07:21
EXTERNAL STANDARD	.000	07:22
BLANK TEST	.000	07:22
EXTERNAL STANDARD	.000	07:23
BLANK TEST	.000	07:23
EXTERNAL STANDARD	.000	07:24
BLANK TEST	.000	07:24

IN = 0  
SIN = 1  
RVD = .0006

Operator Signature 