



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY
BREATH ALCOHOL PROGRAM

INTOX DMT MAINTENANCE REPORT

received 8/11/14-cd

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).
Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.
Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

REVIEWED
By Carol Day at 10:11 am, Sep 05, 2014

INTOX DMT SN 500003	NAME OF AGENCY St. Louis County Police Department	DATE OF INSPECTION 08/05/2014
LOCATION OF INSTRUMENT (STREET AND CITY) Affton-Southwest Precinct 9928 Gravois Road 63123		TIME OF INSPECTION 09:16:48

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

DIAGNOSTIC RECORD

DATE AND TIME <u>08/05/2014 09:16:51</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.9°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>46.8°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS

SIMULATOR STANDARD COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER GUTH LOT # 14030 EXP. DATE 01/20/2016

SIMULATOR TEMP (34°C ± 0.2°C) 34.0 SIMULATOR SN SD2689 SIMULATOR EXP DATE 07/17/2015

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.

0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE

0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE

0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.097	TEST 2: 0.097	TEST 3: 0.097
----------------------	----------------------	----------------------

PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 0	0-.04: 1	.05-.09: 0	.10-.14: 3	.15-.19: 2	OVER .19: 3
--------------------	-----------------	-------------------	-------------------	-------------------	--------------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

INSPECTING OFFICER

SIGNATURE <i>PO D ROSE</i>	PRINT FULL NAME PO D ROSE	
TYPE II PERMIT NUMBER 230253	EXPIRATION DATE 11/12/2015	TELEPHONE NUMBER 314-889-2341

RETURN COMPLETED REPORT TO THE **Breath Alcohol Program, MO Department of Health and Senior Services**
Southeast District Office
2875 James Blvd, Poplar Bluff, MO 63901



GUTH LABORATORIES, INC.

690 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-6470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 14030 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **January 22, 2014**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.1215%** (w/vol) ethyl alcohol. The expiration date for this lot number is **January 20, 2016** at 11:59 PM.

When used in a calibrated Simulator, operating at $34^{\circ}\text{C} \pm .2^{\circ}\text{C}$, this solution will give a breath alcohol analysis instrument reading of $0.100 \text{ g}/210\text{L} \pm 3\%$.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN122211-02 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



Simulator Calibration Report

This calibration report is to certify the alcohol reference simulator listed below has been examined and tested using standards traceable to the National Institute of Standards and Technology (NIST) in accordance with the standards set by the Missouri Department of Health and Senior Services Rules and Regulations:
19 CSR 25-30.051 (4).



Technician Printed Name: Donald D. DeBoard


Technician Signature: *Donald D. DeBoard*

Date: 7-17-2014

Contact: Missouri Safety Center

Breath-Alcohol Instrument Training Program

660-543-4834

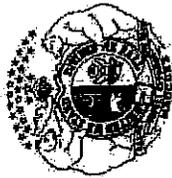

STATE OF MISSOURI
DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM

2

**PERMIT
TYPE II**

DAVID M ROSE

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

DATAMASTER, INTOX DMT, ALCO-SENSOR IV W/PRINTER

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 11/12/2013

NUMBER 230253

EXPIRES 11/12/2015

MO 580-0771 (6-10)

David V. Voshell

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

acting director

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

LAB-4 (R6-10)