

DOH

RECEIVED
By Ellen Strawsine at 1:11 pm, Nov 25, 2015



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY
ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 108271	PRINTER SN 099.3586.813	DATE OF INSPECTION 11/11/2015
LOCATION OF INSTRUMENT (STREET AND CITY) TROOP D, ZONE 3 ZONE OFFICE, BOLIVAR		TIME OF INSPECTION 8:38 am

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

- DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)
- TEMPERATURE OF ALCO SENSOR (10°C - 40°C) 21°C
- PRINTER WORKING PROPERLY
- TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

- SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER Repro Marketing Inc LOT # 14001 EXP. DATE 04/30/2016

SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34.00 SIMULATOR SN MP2146 SIMULATOR EXP DATE 06/09/2016

- CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)
 - 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
 - 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
 - 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 \bullet .103	TEST 2 \bullet .105	TEST 3 \bullet .104
-----------------------	-----------------------	-----------------------

- RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	0	(0-.04)	0	(.05-.09)	0	(.10-.14)	0	(.15-.19)	0	(OVER .19)	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	------------	---

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

INSPECTING OFFICER

SIGNATURE <i>Joshua L White</i>	PRINT NAME JOSHUA L. WHITE
------------------------------------	-------------------------------

TYPE II PERMIT NUMBER/EXPIRATION DATE 240080 03/07/2016	TELEPHONE NUMBER (417) 895-6868
--	------------------------------------

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office
2875 James Boulevard
Poplar Bluff, MO 63901



Missouri Department of Health and Senior Services
 P.O. Box 570, Jefferson City, MO 65102-0570 Phone: 573-751-6400 FAX: 573-751-6010
 RELAY MISSOURI for Hearing and Speech Impaired 1-800-735-2966 VOICE 1-800-735-2466
 Gall Vasterling
 Director



Jeremiah W. (Jay) Nixon
 Governor

Missouri Department of Health and Senior Services Breath Alcohol Program

SIMULATOR CALIBRATION REPORT

This is to certify that the simulator listed below has been examined and tested using standards traceable to the National Institute of Standards and Technology (NIST) in accordance to the standards set by the Rules of Missouri Department of Health and Senior Services, 19 CSR 25-30.

SIMULATOR INFORMATION

Agency: Missouri State Highway Patrol
 Serial Number: MP2146
 Manufacturer: Guth
 Model Number: 12V500

CALIBRATION RESULTS

<u>Reference Temperature</u>	<u>Simulator Temperature</u>
34.00	34.00

This calibration was performed with NIST-Traceable Thermometer SN: 307715

This simulator was tested by: JLC

This testing was performed: 06/09/2015

Signature of certifying DHSS Scientist: 

Name of certifying DHSS Scientist: Brian M. Lutmer

CERTIFICATE OF ANALYSIS

MANUFACTURER AND SUPPLIER: RepCo Marketing, Inc.
LOT NUMBER: 14001
EXPIRATION DATE: April 30, 2016 at 11:59 p.m.

RepCo Marketing, Inc. certifies the following:


RepCo Marketing, Inc. manufactured, tested and supplied Lot Number 14001 of Alcohol Certified Solution for simulators. Random samples of said lot number were analyzed by an independent laboratory utilizing a gas chromatograph and found to contain .1216 gms/dl +/-0.003 gms/dl wt/vol ethanol (95% Confidence).

The alcohol and distilled water used in the solution were found to be free of any interfering substance.

This solution will produce a vapor alcohol value of .100 +/-3% gms/210L Breath when heated to 34 Degrees Celsius +/-0.2 Degrees Celsius in a simulator (95% Confidence).

The date of manufacture for this lot number is May 1, 2014
The expiration date for this lot number is April 30, 2016 at 11:59 p.m.

This document is a true representation of the original Certificate of Analysis.

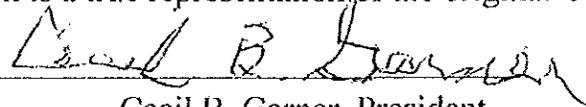

Cecil B. Garner, President
RepCo Marketing, Inc.

AS IV Serial no: 108271
Version no: 532B

TEST RECORD 00094

Temp Date Time ^{s/} 210L

Air Blank:
11/11/15 08:41 .000
Calibration Check:
22 11/11/15 08:41 .104

Subject Name

Subject I.D.

Operator Name, I.D.

TPR. J.L. WHITE 1202

Location

TROOP D ZONE 3
ZONE OFFICE
BOLZUAR, MO

AS IV Serial no: 108271
Version no: 532B

TEST RECORD 00092

Temp Date Time ^{s/} 210L

Air Blank:
11/11/15 08:38 .000
Calibration Check:
21 11/11/15 08:38 .103

Subject Name

Subject I.D.

Operator Name, I.D.

TPR. J.L. WHITE 1202

Location

TROOP D ZONE 3
ZONE OFFICE
BOLZUAR, MO

AS IV Serial no: 108271
Version no: 532B

TEST RECORD 00095

Temp Date Time ^{s/} 210L

VOID: RFI
12 11/11/15 08:42

Subject Name

Subject I.D.

Operator Name, I.D.

TPR. J.L. WHITE 1202

Location

TROOP D ZONE 3
ZONE OFFICE
BOLZUAR, MO

AS IV Serial no: 108271
Version no: 532B

TEST RECORD 00093

Temp Date Time ^{s/} 210L

Air Blank:
11/11/15 08:39 .000
Calibration Check:
21 11/11/15 08:39 .105

Subject Name

Subject I.D.

Operator Name, I.D.

TPR. J.L. WHITE 1202

Location

TROOP D, ZONE 3
ZONE OFFICE
BOLZUAR, MO