



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM

INTOX EC/IR II MAINTENANCE REPORT

RECEIVED

By Carol Day at 7:54 am, Jul 28, 2015

Complete this report at the time of the regular monthly preventive maintenance (30 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12840	NAME OF AGENCY Parkville Police Dept.	DATE OF INSPECTION 07/19/2015
LOCATION OF INSTRUMENT (STREET AND CITY) 8880 NW Clark Avenue Parkville, Missouri		TIME OF INSPECTION 08:14 CDT

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST
<input checked="" type="checkbox"/> ETH CHECK	

BREATH ANALYZER ACCURACY STANDARDS

<input checked="" type="checkbox"/> SIMULATOR SOLUTION	<input type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE
<input checked="" type="checkbox"/> STANDARD SUPPLIER GUTH	LOT# 14220
	EXP. DATE 09/24/2016
<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ±0.2°C) 34°C +/- .2°	SIMULATOR S/N sd2691
	SIMULATOR EXP DATE 10/02/2015

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1 0.100 g/210L	TEST 2 0.100 g/210L	TEST 3 0.101 g/210L
---------------------	---------------------	---------------------

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS	1	0-.04	1	.05-.09	0	.10-.14	0	.15-.19	0	OVER .19	1
----------	---	-------	---	---------	---	---------	---	---------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

INSPECTING OFFICER

SIGNATURE: *Eddie Olson*

TYPE II PERMIT NUMBER: 240375

EXPIRATION DATE: 10/17/2016

PRINT FULL NAME: OLSON, EDWARD

TELEPHONE NUMBER: (816) 741-4454

RETURN COMPLETED REPORT TO THE:
 Breath Alcohol Program, Missouri Department of Health and Senior Services,
 Southeast District Office, 2875 James Blvd, Poplar Bluff, MO 63901

**GUTH LABORATORIES, INC.**

690 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-584-5470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 14220 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on September 25, 2014, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1209% (w/vol) ethyl alcohol. The expiration date for this lot number is September 24, 2016 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

*Testing was conducted using Cerilliant Reference Standard lot number FN08051301 whose values are traceable to NIST.
All balances are calibrated annually by an outside agency using NIST traceable weights.
Calibration verification is done prior to each use utilizing NIST traceable weights.*