



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM
INTOX EC/IR II MAINTENANCE REPORT

RECEIVED

REPORT #3

By Carol Day at 8:52 am, Sep 09, 2015

Complete this report at the time of the regular monthly preventive main (days). Complete this report whenever the instrument is serviced or repaired into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12049	NAME OF AGENCY Willard Police Dept.	DATE OF INSPECTION 09/03/2015
LOCATION OF INSTRUMENT (STREET AND CITY) 795 Hughes Rd Willard		TIME OF INSPECTION 19:30 CDT

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST
<input checked="" type="checkbox"/> ETH CHECK	

BREATH ANALYZER ACCURACY STANDARDS

<input checked="" type="checkbox"/> SIMULATOR SOLUTION	<input type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE	
<input checked="" type="checkbox"/> STANDARD SUPPLIER GUTH	LOT# 15050	EXP. DATE 03/09/2017
<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ±0.2°C) 34°C +/- .2°	SIMULATOR S/N SD2262	SIMULATOR EXP DATE 01/13/2016

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1 @ 0.095 g/210L	TEST 2 @ 0.096 g/210L	TEST 3 @ 0.095 g/210L
-----------------------	-----------------------	-----------------------

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS	1	0-.04	0	.05-.09	1	.10-.14	1	.15-.19	2	OVER .19	0
----------	---	-------	---	---------	---	---------	---	---------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

INSPECTING OFFICER

SIGNATURE 	PRINT FULL NAME ROBERTS, AARON
TYPE 1 PERMIT NUMBER 250079	TELEPHONE NUMBER (417) 742-3077
EXPIRATION DATE 04/29/2017	

RETURN COMPLETED REPORT TO THE:
 Breath Alcohol Program, Missouri Department of Health and Senior Services,
 Southeast District Office, 2875 James Blvd, Poplar Bluff, MO 63901

**GUTH LABORATORIES, INC.**

530 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-584-5470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 15050 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on March 11, 2015, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1218% (w/vol) ethyl alcohol. The expiration date for this lot number is March 9, 2017 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08051301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

STATE OF MISSOURI
DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT
TYPE II

AARON ROBERTS

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

INTOX EC/IR II


for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 505.111 through 505.119, RSMo.

DATE 4/29/2015


NUMBER 250079

EXPIRES 4/29/2017

MO-225.0731(6-0)

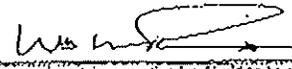

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

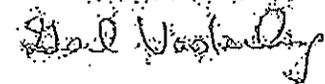

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

LAB-018-101

STATE OF MISSOURI
DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.



Operator ROBERTS, AARON
Permit No 250079
Date Issued 4/29/2015 Date Expires 4/29/2017