



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 STATE PUBLIC HEALTH LABORATORY  
 BREATH ALCOHOL PROGRAM  
**INTOX DMT MAINTENANCE REPORT**

**RECEIVED** REPORT #1  
 By Carol Day at 8:34 am, Mar 19, 2015

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 30 days).  
 Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.  
 Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN <b>500028</b>	NAME OF AGENCY <b>Platte County Sheriff's Office</b>	DATE OF INSPECTION <b>03/18/2015</b>
LOCATION OF INSTRUMENT (STREET AND CITY) <b>415 Third Street, Platte City, Missouri 64079</b>		TIME OF INSPECTION <b>06:43:09</b>

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> <b>DIAGNOSTIC RECORD</b>	
DATE AND TIME <u>03/18/2015 06:43:11</u>	<input checked="" type="checkbox"/> <b>DETECTOR</b>
<input checked="" type="checkbox"/> <b>PROGRAM</b>	<input checked="" type="checkbox"/> <b>FILTER 1</b>
<input checked="" type="checkbox"/> <b>SAMPLE CHAMBER <u>48.8°C</u></b>	<input checked="" type="checkbox"/> <b>FILTER 2</b>
<input checked="" type="checkbox"/> <b>BREATH TUBE <u>45.6°C</u></b>	<input checked="" type="checkbox"/> <b>FILTER 3</b>
<input checked="" type="checkbox"/> <b>PUMP</b>	<input checked="" type="checkbox"/> <b>INTERNAL STANDARD</b>

<b>BREATH ANALYZER ACCURACY STANDARDS</b>	
<input checked="" type="checkbox"/> <b>SIMULATOR STANDARD</b>	<input type="checkbox"/> <b>COMPRESSED ETHANOL-GAS MIXTURE</b>

<input checked="" type="checkbox"/> <b>STANDARD SUPPLIER <u>GUTH</u></b>	<b>LOT # <u>14220</u></b>	<b>EXP. DATE <u>09/24/2016</u></b>
--	---------------------------	------------------------------------

<input checked="" type="checkbox"/> <b>SIMULATOR TEMP (34°C ± 0.2°C) <u>34.0</u></b>	<b>SIMULATOR SN <u>SD2780</u></b>	<b>SIMULATOR EXP DATE <u>07/15/2015</u></b>
--	-----------------------------------	---

<input checked="" type="checkbox"/> <b>CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)</b> Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.	
<input checked="" type="checkbox"/> <b>0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE</b>	
<input type="checkbox"/> <b>0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE</b>	
<input type="checkbox"/> <b>0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE</b>	

<b>TEST 1: 0.100</b>	<b>TEST 2: 0.099</b>	<b>TEST 3: 0.100</b>
----------------------	----------------------	----------------------

<input checked="" type="checkbox"/> <b>PERFORM R.F.I. TEST</b>
--

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

<b>REFUSALS: 0</b>	<b>0-.04: 71</b>	<b>.05-.09: 3</b>	<b>.10-.14: 2</b>	<b>.15-.19: 5</b>	<b>OVER .19: 5</b>
--------------------	------------------	-------------------	-------------------	-------------------	--------------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

---



---



---



---

<b>INSPECTING OFFICER</b>		
SIGNATURE 	PRINT FULL NAME <b>MEGHAN M BLACKMORE</b>	
TYPE II PERMIT NUMBER <b>240404</b>	EXPIRATION DATE <b>11/20/2016</b>	TELEPHONE NUMBER <b>816-858-2424</b>

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services  
 Southeast District Office  
 2875 James Blvd, Poplar Bluff, MO 63901



## GUTH LABORATORIES, INC.

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

### CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number **14220** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **September 25, 2014**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.1209%** (w/vol) ethyl alcohol. The expiration date for this lot number is **September 24, 2016** at 11:59 PM.

When used in a calibrated Simulator, operating at  $34^{\circ}\text{C} \pm .2^{\circ}\text{C}$ , this solution will give a breath alcohol analysis instrument reading of **0.100 g/210L  $\pm$  3%**.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President  
GUTH LABORATORIES, INC.

*NIST Traceability:*

*Testing was conducted using Cerilliant Reference Standard lot number FN08051301 whose values are traceable to NIST.*

*All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.*



STATE OF MISSOURI  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM



**PERMIT**  
**TYPE III**  
**MEGHAN M BLACKMORE**

is hereby authorized to operate the following breath analyzer(s):

**ALCO-SENSOR IV WITH PRINTER, DATAMASTER**

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 6/13/2014

NUMBER 342568

EXPIRES 6/13/2016

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

, director

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MO 580-0772 (6-10)

LAB-6 (R6-10)

**STATE OF MISSOURI**  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM

**INSTRUMENT OPERATOR CARD**

*The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.*

Operator BLACKMORE, MEGHAN  
 Permit No 342568  
 Date Issued 6/13/2014 Date Expires 6/13/2016