



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY
BREATH ALCOHOL PROGRAM

INTOX DMT MAINTENANCE REPORT

RECEIVED

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).
Complete this report whenever the instrument is serviced or repaired and when
Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.
By Carol Day at 8:57 am, Aug 25, 2015

INTOX DMT SN 500258	NAME OF AGENCY Mountain Grove Police Department	DATE OF INSPECTION 08/21/2015
------------------------	--	----------------------------------

LOCATION OF INSTRUMENT (STREET AND CITY) Mountain Grove Police 102 East State Street	TIME OF INSPECTION 11:22:22
---	--------------------------------

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
DATE AND TIME <u>08/21/2015 11:22:24</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.8°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>46.9°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS	
<input checked="" type="checkbox"/> SIMULATOR STANDARD	<input type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE

<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>GUTH</u>	LOT # <u>14220</u>	EXP. DATE <u>09/24/2016</u>
<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) <u>34.0</u>	SIMULATOR SN <u>SD2171</u>	SIMULATOR EXP DATE <u>12/16/2015</u>

<input checked="" type="checkbox"/> CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT) Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.
<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.101	TEST 2: 0.100	TEST 3: 0.100
---------------	---------------	---------------

<input checked="" type="checkbox"/> PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 0	0-.04: 0	.05-.09: 0	.10-.14: 1	.15-.19: 2	OVER .19: 0
-------------	----------	------------	------------	------------	-------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

INSPECTING OFFICER

SIGNATURE <i>Daniel R Bledsoe</i>	PRINT FULL NAME DANIEL R BLEDSOE
--------------------------------------	-------------------------------------

TYPE II PERMIT NUMBER 240433	EXPIRATION DATE 12/17/2016	TELEPHONE NUMBER 417-926-5181
---------------------------------	-------------------------------	----------------------------------

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services
Southeast District Office
2875 James Blvd, Poplar Bluff, MO 63901

**GUTH LABORATORIES, INC.**

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-6470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 14220 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on September 25, 2014, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1209% (w/vol) ethyl alcohol. The expiration date for this lot number is September 24, 2016 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.


NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08051301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI
DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM

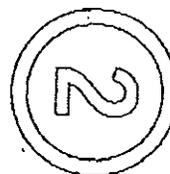

PERMIT
TYPE II

DANIEL BLEDSOE

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

DATAMASTER, INTOX DMT

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 12/17/2014

NUMBER 240433

EXPIRES 12/17/2016

MO-SSO-0771 (6-10)

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

acting director
DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

LAB-4 Pre-101