

**RECEIVED**

By Brian Lutmer at 10:51 am, Jun 08, 2016



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
STATE PUBLIC HEALTH LABORATORY  
BREATH ALCOHOL PROGRAM

**INTOX EC/IR II MAINTENANCE REPORT**

REPORT #3

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12945	NAME OF AGENCY St. Joseph Police	DATE OF INSPECTION 06/04/2016
LOCATION OF INSTRUMENT (STREET AND CITY) 501 Faraon Street St. Joseph		TIME OF INSPECTION 15:35 CDT

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

**DIAGNOSTIC RECORD**

<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST
<input checked="" type="checkbox"/> ETH CHECK	

**BREATH ANALYZER ACCURACY STANDARDS**

<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE		
<input checked="" type="checkbox"/> STANDARD SUPPLIER	INTOXIMETERS	LOT# AG509101	EXP. DATE 04/01/2017
<input type="checkbox"/> SIMULATOR TEMP (34°C ±0.2°C)	SIMULATOR S/N	SIMULATOR EXP DATE	

**CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)**

Run three tests using a standard solution. All three tests must be within +5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

- 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
- 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
- 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1 → 0.099 g/210L	TEST 2 → 0.099 g/210L	TEST 3 → 0.099 g/210L
-----------------------	-----------------------	-----------------------

**INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:**

REFUSALS	1	0-.04	3	.05-.09	1	.10-.14	4	.15-.19	3	OVER .19	3
----------	---	-------	---	---------	---	---------	---	---------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

June Maintenance

INSPECTING OFFICER		PRINT FULL NAME	
SIGNATURE		GARY, SCOTT	
TYPE II PERMIT NUMBER	EXPIRATION DATE	TELEPHONE NUMBER	
250127	06/08/2017	( 816 ) 271-5359	

**RETURN COMPLETED REPORT TO THE:**

Breath Alcohol Program, Missouri Department of Health and Senior Services,  
Southeast District Office, 2875 James Blvd, Poplar Bluff, MO 63901



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 STATE PUBLIC HEALTH LABORATORY  
 BREATH ALCOHOL PROGRAM

**INTOX EC/IR II MAINTENANCE REPORT**

REPORT #3

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12947	NAME OF AGENCY ST. JOSEPH POLICE DEPT.	DATE OF INSPECTION 06/04/2016
----------------------------	---	----------------------------------

LOCATION OF INSTRUMENT (STREET AND CITY) 501 FARAON ST. JOSEPH	TIME OF INSPECTION 15:37 CDT
---	---------------------------------

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> <b>DIAGNOSTIC RECORD</b>	
<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST
<input checked="" type="checkbox"/> ETH CHECK	

**BREATH ANALYZER ACCURACY STANDARDS**

<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE				
<input checked="" type="checkbox"/> STANDARD SUPPLIER	INTOXIMETERS	LOT#	AG509101	EXP. DATE	04/01/2017
<input type="checkbox"/> SIMULATOR TEMP (34°C ±0.2°C)	SIMULATOR S/N	SIMULATOR EXP DATE			

**CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)**

Run three tests using a standard solution. All three tests must be within +5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1	0.099 g/210L	TEST 2	0.099 g/210L	TEST 3	0.099 g/210L
--------	--------------	--------	--------------	--------	--------------

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS	0	0-.04	0	.05-.09	1	.10-.14	0	.15-.19	0	OVER .19	0
----------	---	-------	---	---------	---	---------	---	---------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

June Maintenance

INSPECTING OFFICER	
SIGNATURE	PRINT FULL NAME
	GARY, SCOTT
TYPE II PERMIT NUMBER	TELEPHONE NUMBER
250127	( 816 ) 271-5359
EXPIRATION DATE	
06/08/2017	

**RETURN COMPLETED REPORT TO THE:**  
 Breath Alcohol Program, Missouri Department of Health and Senior Services,  
 Southeast District Office, 2875 James Blvd, Poplar Bluff, MO 63901





STATE OF MISSOURI  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM

2

**PERMIT**  
**TYPE II**  
**SCOTT GARY**

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

**DATAMASTER, INTOX EC/IR II, ALCO-SENSOR IV W/PRINTER**

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 6/8/2015

NUMBER 250127

EXPIRES 6/8/2017

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MO 580-0771 (6-10)

LAB 4 (R5-10)



**STATE OF MISSOURI**  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM

**INSTRUMENT OPERATOR CARD**

*The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.*



Operator **GARY, SCOTT**  
 Permit No **250127**  
 Date Issued **6/8/2015**    Date Expires **6/8/2017**