



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
STATE PUBLIC HEALTH LABORATORY  
BREATH ALCOHOL PROGRAM

**INTOX DMT MAINTENANCE REPORT**

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).  
Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.  
Retain the original and send a copy within 5 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN <b>500007</b>	NAME OF AGENCY <b>St Louis County Intake</b>	DATE OF INSPECTION <b>02/02/2016</b>
LOCATION OF INSTRUMENT (STREET AND CITY) <b>100 S. Central, Clayton MO</b>		TIME OF INSPECTION <b>12:14:49</b>

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
DATE AND TIME <u>02/02/2016 12:14:51</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.8°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>48.1°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS	
<input checked="" type="checkbox"/> SIMULATOR STANDARD	<input type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE

<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>GUTH</u>	LOT # <u>15220</u>	EXP. DATE <u>09/28/2017</u>
---	--------------------	-----------------------------

<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) <u>34.0</u>	SIMULATOR SN <u>SD2671</u>	SIMULATOR EXP DATE <u>07/15/2016</u>
---	----------------------------	--------------------------------------

<input checked="" type="checkbox"/> CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT) Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.	
<input checked="" type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE	
<input type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE	
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE	

TEST 1: <b>0.100</b>	TEST 2: <b>0.100</b>	TEST 3: <b>0.100</b>
----------------------	----------------------	----------------------

<input checked="" type="checkbox"/> PERFORM R.F.I. TEST
---

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: <b>1</b>	0-.04: <b>0</b>	.05-.09: <b>2</b>	.10-.14: <b>3</b>	.15-.19: <b>2</b>	OVER .19: <b>4</b>
--------------------	-----------------	-------------------	-------------------	-------------------	--------------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY):


**INSPECTING OFFICER**

SIGNATURE 	PRINT FULL NAME <b>KEVIN R VILMER</b>
---------------	--

TYPE II PERMIT NUMBER <b>250240</b>	EXPIRATION DATE <b>10/30/2017</b>	TELEPHONE NUMBER <b>314-615-7101</b>
--	--------------------------------------	---

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services  
Southeast District Office  
2875 James Blvd, Poplar Bluff, MO 63901



Missouri Department of Health and Senior Services  
 P.O. Box 570, Jefferson City, MO 65102-0570 Phone: 573-751-6400 FAX: 573-751-6010  
 RELAY MISSOURI for Hearing and Speech Impaired 1-800-735-2966 VOICE 1-800-735-2466  
 Gail Vasterling  
 Director



Jeremiah W. (Jay) Nixon  
 Governor

Missouri Department of Health and Senior Services Breath Alcohol Program

***SIMULATOR CALIBRATION REPORT***

This is to certify that the simulator listed below has been examined and tested using standards traceable to the National Institute of Standards and Technology (NIST) in accordance to the standards set by the Rules of Missouri Department of Health and Senior Services, 19 CSR 25-30.

**SIMULATOR INFORMATION**

Agency: St. Louis County Sheriff Department  
 Serial Number: SD2671  
 Manufacturer: Guth  
 Model Number: 10-4D

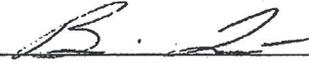
**CALIBRATION RESULTS**

<u>Reference Temperature</u>	<u>Simulator Temperature</u>
34.00	34.0

This calibration was performed with NIST-Traceable Thermometer SN: 094948

This simulator was tested by: DRL

This testing was performed: 07/15/2015

Signature of certifying DHSS Scientist: 

Name of certifying DHSS Scientist: Brian M. Lutmer



STATE OF MISSOURI  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM

2

**PERMIT**  
**TYPE II**  
**KEVIN R VILMER**

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

**INTOX DMT**

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 10/30/2015

NUMBER 250240

EXPIRES 10/30/2017

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MO 580-0771 (6-10)

LAB-4 (R6-10)

STATE OF MISSOURI  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM

**INSTRUMENT OPERATOR CARD**

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.

Operator VILMER, KEVIN  
 Permit No 250240  
 Date Issued 10/30/2015 Date Expires 10/30/2017



## GUTH LABORATORIES, INC.

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

### CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number **15220** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **September 30, 2015**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.1214%** (w/vol) ethyl alcohol. The expiration date for this lot number is **September 28, 2017** at 11:59 PM.

When used in a calibrated Simulator, operating at  $34^{\circ}\text{C} \pm .2^{\circ}\text{C}$ , this solution will give a breath alcohol analysis instrument reading of **0.100 g/210L  $\pm$  3%**.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President  
GUTH LABORATORIES, INC.

**NIST Traceability:**

Testing was conducted using Cerilliant Reference Standard lot number **FN08051301** whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.