



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
 STATE PUBLIC HEALTH LABORATORY
 BREATH ALCOHOL PROGRAM
 INTOX DMT MAINTENANCE REPORT

RECEIVED

By Carol Day at 8:10 am, May 12, 2016

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance.
 Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.
 Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN 500013	NAME OF AGENCY Grandview Police Department	DATE OF INSPECTION 05/09/2016
LOCATION OF INSTRUMENT (STREET AND CITY) 1200 Main Street Grandview, MO 64030		TIME OF INSPECTION 15:35:15

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
DATE AND TIME <u>05/09/2016 15:35:17</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.7°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>46.6°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS		
<input checked="" type="checkbox"/> SIMULATOR STANDARD	<input type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE	
<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>GUTH</u>	LOT # <u>16040</u>	EXP. DATE <u>01/20/2018</u>
<input checked="" type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) <u>34.0</u>	SIMULATOR SN <u>DR5385</u>	SIMULATOR EXP DATE <u>07/14/2016</u>

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
 Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.

0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.097	TEST 2: 0.098	TEST 3: 0.098
---------------	---------------	---------------

PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 0	0-.04: 0	.05-.09: 0	10- 14: 0	.15- .19: 0	OVER 19: 0
-------------	----------	------------	-----------	-------------	------------


LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

INSPECTING OFFICER

SIGNATURE 	PRINT FULL NAME BRANDON P GRANTHAM
---------------	---------------------------------------

TYPE II PERMIT NUMBER 250207	EXPIRATION DATE 10/01/2017	TELEPHONE NUMBER 816-316-4912
---------------------------------	-------------------------------	----------------------------------

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services
 Southeast District Office
 2875 James Blvd, Poplar Bluff, MO 63901

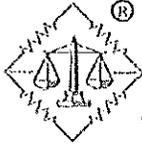

GUTH LABORATORIES, INC.

580 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 16040 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on January 22, 2016, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1213% (w/vol) ethyl alcohol. The expiration date for this lot number is January 20, 2018 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN08051301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

2

PERMIT
 TYPE II

BRANDON P GRANTHAM

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER, INTOX DMT

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 10/1/2015

NUMBER 250207

EXPIRES 10/1/2017

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MSL 237 02-1 (6-10)

LAB-4 (116-10)

STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.

Operator GRANTHAM, BRANDON
 Permit No 250207
 Date Issued 10/1/2015 Date Expires 10/1/2017