

RECEIVED

By Carol Day at 4:02 pm, Mar 31, 2016



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
STATE PUBLIC HEALTH LABORATORY  
BREATH ALCOHOL PROGRAM

INTOX DMT MAINTENANCE REPORT

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).  
Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.  
Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN 500105	NAME OF AGENCY Missouri State Highway Patrol	DATE OF INSPECTION 03/30/2016
LOCATION OF INSTRUMENT (STREET AND CITY) Wayne Co. SO, 100 Maple St., Greenville		TIME OF INSPECTION 12:18:11

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

DIAGNOSTIC RECORD

DATE AND TIME <u>03/30/2016 12:18:13</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.8°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>48.1°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS

SIMULATOR STANDARD  COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER ILMO LOT # 25814080A4 EXP. DATE 10/05/2016

SIMULATOR TEMP (34°C ± 0.2°C) \_\_\_\_\_ SIMULATOR SN \_\_\_\_\_ SIMULATOR EXP DATE \_\_\_\_\_

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)  
Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.

0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE

0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE

0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.077      TEST 2: 0.077      TEST 3: 0.077

PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 0	0-.04: 0	.05-.09: 0	.10-.14: 0	.15-.19: 1	OVER .19: 0
-------------	----------	------------	------------	------------	-------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

---



---



---



---



---

INSPECTING OFFICER

SIGNATURE 	PRINT FULL NAME JOHN J WEST	
TYPE II PERMIT NUMBER 240065	EXPIRATION DATE 02/04/2018	TELEPHONE NUMBER 573-840-9500

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services  
Southeast District Office  
2875 James Blvd, Poplar Bluff, MO 63901



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790  
217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

## Certificate of Analysis

**Certificate ID:** 7200  
**Part #:** BAC108L080T  
**Cylinder Size:** 108L  
**Lot Number:** 25814080A4  
**Expiration:** 10/5/2016

**0.080 BAC** (For the calibration of instruments used to determine breath alcohol concentration)

**Contents:** 108 Liters @ 1200 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	208 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	Balance	BAC whichever is greater	

\*NIST Standard Reference Material  
Cylinder No. CCI4290 / Job No. 09160202  
Certified 212.8 µmol/mol Ethanol in Nitrogen  
for ILMO Products Co., Jacksonville, IL

Store in dry area, away from sources of heat, ignition  
and direct sunlight. Do not allow storage area to  
exceed 52 °C (125 °F).

  
Specialty Gas Lab Tech

09/25/14  
Date

Distributed by: CMI Inc.  
316 East Ninth Street  
Owensboro, KY 42303  
Phone 866-835-0690  
[www.alcoholtest.com](http://www.alcoholtest.com)



ISO/IEC 17025:2005 Accredited Laboratory