



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY
BREATH ALCOHOL PROGRAM

INTOX DMT MAINTENANCE REPORT

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).
Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.
Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN 500150	NAME OF AGENCY Missouri State Highway Patrol	DATE OF INSPECTION 02/17/2016
LOCATION OF INSTRUMENT (STREET AND CITY) 104 W. Main St., Warrenton 63383		TIME OF INSPECTION 22:01:33

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
DATE AND TIME <u>02/17/2016 22:01:35</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.8°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>47.2°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS	
<input type="checkbox"/> SIMULATOR STANDARD	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE

<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>ILMO</u>	LOT # <u>25814080A3</u>	EXP. DATE <u>10/05/2016</u>
---	-------------------------	-----------------------------

<input type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) _____	SIMULATOR SN _____	SIMULATOR EXP DATE _____
--	--------------------	--------------------------

<input checked="" type="checkbox"/> CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT) Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.	
<input type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE	
<input checked="" type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE	
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE	

TEST 1: 0.077	TEST 2: 0.076	TEST 3: 0.077
---------------	---------------	---------------

<input checked="" type="checkbox"/> PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 0	0-04: 0	05-09: 0	10-14: 2	15-19: 3	OVER 19: 0
-------------	---------	----------	----------	----------	------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

INSPECTING OFFICER

SIGNATURE 	PRINT FULL NAME JEFFREY S GRAUE
---------------	---

TYPE II PERMIT NUMBER 260026	EXPIRATION DATE 01/29/2018	TELEPHONE NUMBER 636-300-2800
--	--------------------------------------	---

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services
Southeast District Office
2875 James Blvd, Poplar Bluff, MO 63901



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790
217-245-2183 • Fax: 217-243-7634 • www.ilmoproducts.com

Certificate of Analysis

Certificate ID: 7199
Part #: BAC108L080T
Cylinder Size: 108L
Lot Number: 25814080A3
Expiration: 10/5/2016

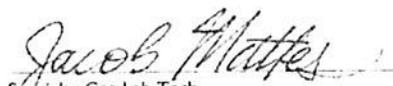
0.080 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents: 108 Liters @ 1200 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	208 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	balance	BAC whichever is greater	

*NIST Standard Reference Material
Cylinder No. CCI4290 / Job No. 09160202
Certified 212.8 µmol/mol Ethanol in Nitrogen
for ILMO Products Co., Jacksonville, IL

Store in dry area, away from sources of heat, ignition
and direct sunlight. Do not allow storage area to
exceed 52 °C (125 °F).


Specialty Gas Lab Tech

09/25/14
Date

Distributed by: CMI Inc.
316 East Ninth Street
Owensboro, KY 42303
Phone 866-835-0690
www.alcoholtest.com





STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

2

PERMIT
TYPE II
JEFFREY S GRAUE

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

INTOX DMT

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 1/29/2016

NUMBER 260026

EXPIRES 1/29/2018

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.

Operator GRAUE, JEFFREY
 Permit No 260026
 Date Issued 1/29/2016 Date Expires 1/29/2018