



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY
BREATH ALCOHOL PROGRAM

RECEIVED

By Ellen Strawsine at 10:43 am, Jan 07, 2016

INTOX DMT MAINTENANCE REPORT

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).
Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.
Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN 500179	NAME OF AGENCY Missouri State Highway Patrol	DATE OF INSPECTION 01/01/2016
LOCATION OF INSTRUMENT (STREET AND CITY) 209 SE 2nd Ave., Ava, MO 65608		TIME OF INSPECTION 09:08:24

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIAGNOSTIC RECORD	
DATE AND TIME <u>01/01/2016 09:08:26</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.7°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>45.0°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

BREATH ANALYZER ACCURACY STANDARDS	
<input type="checkbox"/> SIMULATOR STANDARD	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE
<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>ILMO</u>	LOT # <u>25814080A3</u> EXP. DATE <u>10/05/2016</u>
<input type="checkbox"/> SIMULATOR TEMP (34°C ± 0.2°C) _____	SIMULATOR SN _____ SIMULATOR EXP DATE _____

<input checked="" type="checkbox"/> CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT) Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.
<input type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input checked="" type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.077	TEST 2: 0.077	TEST 3: 0.077
---------------	---------------	---------------

PERFORM R.F.I. TEST

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:

REFUSALS: 1	0-.04: 1	.05-.09: 0	.10-.14: 0	.15-.19: 1	OVER .19: 0
-------------	----------	------------	------------	------------	-------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

THIS INSTRUMENT IS OPERATING WITHIN THE GUIDELINES SET BY THE MISSOURI DEPARTMENT OF HEALTH.

INSPECTING OFFICER	
SIGNATURE 	PRINT FULL NAME D J JOHNSON
TYPE II PERMIT NUMBER 240172	EXPIRATION DATE 04/22/2016
	TELEPHONE NUMBER 417-469-3121

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, MO Department of Health and Senior Services
Southeast District Office
2875 James Blvd, Poplar Bluff, MO 63901



7 Eastgate Dr. • P.O. Box 790 • Jacksonville, IL 62651-0790
217-245-2183 • Fax: 217-243-7634 • www.ilmo-products.com

Certificate of Analysis

Certificate ID: 7199
Part #: BAC108L080T
Cylinder Size: 108L
Lot Number: 25814080A3
Expiration: 10/5/2016

0.080 BAC (For the calibration of instruments used to determine breath alcohol concentration)

Contents: 108 Liters @ 1200 psig 70°F (21°C)

Component:	Concentration:	Accuracy:	Method:
Ethanol	208 ppm	+/- 0.002 or 2%	NDIR
Nitrogen	balance	BAC whichever is greater	

*NIST Standard Reference Material
Cylinder No. CCI-4290 / Job No. 09160202
Certified 212.8 µmol/mol Ethanol in Nitrogen
for ILMO Products Co., Jacksonville, IL

Store in dry area, away from sources of heat, ignition and direct sunlight. Do not allow storage area to exceed 52 °C (125 °F).

Jacob Witter
Specialty Gas Lab Tech

09/25/14
Date

Distributed by:

CMI Inc.
316 East Ninth Street
Owensboro, KY 42303
Phone 866-835-0690
www.alcoholtest.com



ISO/IEC 17025:2005 Accredited Laboratory



STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

2

**PERMIT
 TYPE II**

DANIEL J JOHNSON

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

DATAMASTER, INTOX DMT

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 4/22/2014

NUMBER 240172

EXPIRES 4/22/2016

MO 590-0771 (6-10)

[Signature]

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

[Signature]

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

LAB-4 (RS-10)