



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 STATE PUBLIC HEALTH LABORATORY  
**ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT**

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 097423	NAME OF AGENCY MSHP	DATE OF INSPECTION 11/01/2023
LOCATION OF INSTRUMENT (STREET AND CITY) 891 Technology Drive, Weldon Spring		TIME OF INSPECTION 11:50 am

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)

TEMPERATURE OF ALCO SENSOR (10°C - 40°C)

PRINTER WORKING PROPERLY

TIME AND DATE DISPLAYING PROPERLY

**BREATH ALCOHOL ACCURACY STANDARDS**

SIMULATOR SOLUTION  COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER RepCo Marketing Inc. LOT # 23180 EXP. DATE 05/17/2025

SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34.00 SIM. SN MP2223 SIM. NIST EXP DATE 01/27/2024

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

- 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
- 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
- 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 ← .103

TEST 2 ← .101

TEST 3 ← .100

RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:  
 (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	(0-.04)	(.05-.09)	(.10-.14)	(.15-.19)	(OVER .19)
----------	---------	-----------	-----------	-----------	------------

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

**INSPECTING OFFICER**

SIGNATURE

PRINT NAME  
 Dustin J. Duffie

TYPE II PERMIT NUMBER/EXPIRATION DATE  
 2103067 12/14/2023

TELEPHONE NUMBER  
 (636) 300-2800

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office by mail, fax, or email.

AS IV Serial no: 097423  
Version no: 532B

TEST RECORD 00366

Temp Date Time 210L <sup>s/</sup>

Air Blank:  
11/01/23 12:05 .000  
Calibration Check:  
20 11/01/23 12:05 .103

Subject Name  
Maint Test #1  
Subject I.D.

Operator Name, I.D.  
Duffie #1325

Location  
Troop C HQ

AS IV Serial no: 097423  
Version no: 532B

TEST RECORD 00368

Temp Date Time 210L <sup>s/</sup>

Air Blank:  
11/01/23 12:13 .000  
Calibration Check:  
23 11/01/23 12:13 .100

Subject Name  
Maint Test #3  
Subject J.I.

Operator Name, I.D.  
Duffie #1325

Location  
Troop C HQ

AS IV Serial no: 097423  
Version no: 532B

TEST RECORD 00367

Temp Date Time 210L <sup>s/</sup>

Air Blank:  
11/01/23 12:11 .000  
Calibration Check:  
22 11/01/23 12:11 .101

Subject Name  
Maint Test #2  
Subject I.D.

Operator Name, I.D.  
Duffie #1325

Location  
Troop C HQ

AS IV Serial no: 097423  
Version no: 532B

TEST RECORD 00369

Temp Date Time 210L <sup>s/</sup>

VOID: RFI  
12 11/01/23 12:14

Subject Name  
RFI Test  
Subject I.D.

Operator Name, I.D.

Duffie #1325

Location  
Troop C HQ



**GUTH LABORATORIES, INC.**

590 NORTH 67th STREET • HARRISBURG, PA 17111-4811 • TELEPHONE: 717-564-5470

## CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 23180 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on May 18, 2023, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1220% (w/vol) ethyl alcohol. The expiration date for this lot number is May 17, 2025 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President  
GUTH LABORATORIES, INC.

***NIST Traceability:***

*Testing was conducted using Cerilliant Reference Standard lot number FN11172002 whose values are traceable to NIST.*

*All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.*



STATE OF MISSOURI  
DEPARTMENT OF HEALTH AND SENIOR SERVICES  
BREATH ALCOHOL PROGRAM



**PERMIT  
TYPE II**

**DUSTIN J. DUFFIE**

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

**ALCO-SENSOR IV WITH PRINTER, INTOX DMT**

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 12/14/2021

NUMBER 210306

EXPIRES 12/14/2023

*Laura A. Way*

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

*Donald A. Ramsey*

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MO 580-0771 (6-10)

LAB-4 (R6-10)



**STATE OF MISSOURI**  
DEPARTMENT OF HEALTH AND SENIOR SERVICES  
BREATH ALCOHOL PROGRAM

**INSTRUMENT OPERATOR CARD**

*The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.*

**Operator** DUFFIE, DUSTIN  
**Permit No** 210306  
**Date Issued** 12/14/2021    **Date Expires** 12/14/2023

