



RECEIVED

By Tracy Crews at 4:26 pm, Mar 11, 2020

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 097425	PRINTER SN 096.3580.866	DATE OF INSPECTION 02/21/2020
LOCATION OF INSTRUMENT (STREET AND CITY) 14301 SOUTH OUTER 40, CHESTERFIELD		TIME OF INSPECTION 11:12 am

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)
<input checked="" type="checkbox"/> TEMPERATURE OF ALCO SENSOR (10°C - 40°C)
<input checked="" type="checkbox"/> PRINTER WORKING PROPERLY
<input checked="" type="checkbox"/> TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE
<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>INTOXIMETERS, INC.</u>	LOT # <u>AG829708</u> EXP. DATE <u>10/24/2020</u>
<input type="checkbox"/> SIMULATOR TEMPERATURE (34°C ± 0.2°C) _____	SIMULATOR SN _____ SIMULATOR EXP DATE _____

CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

<input checked="" type="checkbox"/> 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
<input type="checkbox"/> 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
<input type="checkbox"/> 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 ● 0.102	TEST 2 ● 0.101	TEST 3 ● 0.102
----------------	----------------	----------------

RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	0	(0-.04)	0	(.05-.09)	0	(.10-.14)	1	(.15-.19)	0	(OVER .19)	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	------------	---

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

PRINTER RIBBON REPLACED

INSPECTING OFFICER	
SIGNATURE 	PRINT NAME PO JAHNS, DSN 3725
TYPE II PERMIT NUMBER/EXPIRATION DATE 290039 02/20/2021	TELEPHONE NUMBER (636) 529-8210

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office
2875 James Boulevard
Poplar Bluff, MO 63901