





MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report Send copy to Departr					nd whenever instrument is repaired.	
ALCO SENSOR IV SN		NAME OF AGENCY HENRY CO SO			DATE OF INSPECTION	
LOCATION OF INSTRUME りがく ないれ ト CHECKLIST: Place a where determined.) U	mark in the box by ea	chitem if found to be	m& \satisfactory or if		TIME OF INSPECTION 13:31 shed limits. (Write in observed values	
DIGITAL READO	UT (ALL ELEMENTS	OPERATIONAL)				
TEMPERATURE	OF ALCO SENSOR	(10°C - 40°C)				
PRINTER WORK	ING PROPERLY			11.000		
TIME AND DATE	DISPLAYING PROP	ERLY				
BREATH ALCOHOL	ACCURACY STAND	ARDS				
SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE						
STANDARD SUP	PLIER GUFL		LOT# _2 #	40 EXP. DAT	E 4-6-907	
SIMULATOR TEN	MPERATURE (34°C ±	0.2°C) 34.00	_ SIM. SN _ 🗠	10)116 SIM.	NIST EXP DATE 1-6-2007	
0.080% STA	NDARD - MUST REA NDARD - MUST REA NDARD - MUST REA	D BETWEEN 0.076%	6 and 0.084% IN	CLUSIVE		
TEST 1 . 100		TEST 2 . 094		TEST 3 🖝	TEST 3 * . 100	
RFI DETECTOR C	PERATING					
INDICATE THE NUME (DO NOT INCLUDE S			WING RANGES	SINCE THE LAST MA	INTENANCE REPORT:	
REFUSALS	(004)	(.0509)	(.1014)	(.1519)	(OVER .19)	
established limits (use			hat was made to	restore the instrument	to operate satisfactorily and within	
	·					
NSPECTING OFFICE	R					
SIGNATURE A CONTROL OF THE CONTROL O				PRINT NAME	MATT BONE	
TYPE II PERMIT NUMBER/EXPIRATION DATE				TELEPHONE NUMBER		
290214	9-14-2021				3- 4597	
Return completed rep		lcohol Program, MO fax, or email.	Department of F	lealth and Senior Servi	ces, Southeast District Office	

MS 19 Serial per 697458 AS IV Serial ned 097459 Versien red 0323 AS 10 Deriot not 697459 Version are 5320 Version not 1878 TENT RECERT SESSE TEST SPEED SEED TOTAL BARRA SERVE Tem: Odi: Timo 2181 Tome Add Time 2181 Air Stank Tome Ada Time 219L And the second s 669/22/26 ER 29 . 666 Air Blank Air Albert 988. ICHIL PYZYYY Californi ione Californion Checks - M9/77/77M 【GECERT , 988 23 40/22/24 10020 100 24 97/27/26 10/01 (188 Calibration Check! -94 MW/VV/58 100 BC . 899 Sulpha to Hames SubJack Rage Calilrita September 1 House TEST Silliates & J. H. TEST Sulphort F. H. AAH Sport of Hopes 1. U. Open dos Homes falla MATT BOY 290211 MATT BIN 250214 Oranida Rages Libra Lemini (1911 9-14, 2021 MATT Bad 290214 Jacob July 4-14-2021 MSC Lancas Jens 1-19-20/1 m sc MEC AS IV CAT OF 100: 897459 AS JV North and 897459 Version sec sous 48 JV Chr Lit Hed 897459 Version and 1227 Version not tropp TERR STRUKT FRANCE THU WILLIAM BEEFS 9/ TERR RELEASE SESSES Temp Hills Time 2181 Term Hale Firm 2181 The state of the s entropy of the second s Tome the Time 2181 Air Allente Air Wlank #2777770 DESERT .000 MSZZZZZB EDDIZ , 688 Varan wh Calleding Charles Scholar Credit Addis 12 M07/27/20 Tables 25 F2222 TEES , 188 26 1977 7779 118 07 (888) Schried Hage

Tomic Hale There 2181
Air Mante Halvey on Livery , 808
Calliferation Charles , 188
Substitut Mante The Substitute of the Substit of the Substitute of the Substitute of the Substitute of the Su

Schrift Hamps

TEST

Schrift I I II

RFI

Check that Hamps I, II,

MATT B.~/ 250214

Length Line 4719-2021

MSC

Bir Hank

HEZYZZYA DECIZ .008

Subder I Tead I Autor

ZG HEZZZZYA EDECIZ .008

Subder I Mane

Blank

Subder I Mane

TEST

Overal on Maney I. II.

MATT Book 299214

Land dinny 1-18-2001

Macc



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 20190 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on April 8, 2020, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1206% (w/vol) ethyl alcohol. The expiration date for this lot number is April 6, 2022 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN02271802 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT

TYPE II MATT B BOND

and operate the following breath analyzer(s): is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs,

INTOX DMT, INTOXILYZER 8000, INTOX EC/IR II, ASIV W/PRINTER

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

We have

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MO 580-0771 (6-10)

EXPIRES 9/19/2021

NUMBER 290214

DATE

9/19/2019

LAB-4, (R6-10)