



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES.
STATE PUBLIC HEALTH LABORATORY
ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 102469	PRINTER SN 095.353.193	DATE OF INSPECTION 10/01/2020
LOCATION OF INSTRUMENT (STREET AND CITY) Lafayette County Sheriff 107 S. 11th Lexington, Mo. 64067		TIME OF INSPECTION 10:38 am

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

- DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)
- TEMPERATURE OF ALCO SENSOR (10°C - 40°C)
- PRINTER WORKING PROPERLY
- TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

- SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE
- STANDARD SUPPLIER Guth Labs LOT # 20190 EXP. DATE 04/06/2022
- SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34.0 SIMULATOR SN SD2275 SIMULATOR EXP DATE 01/07/2021
- CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)
 - 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
 - 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
 - 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 ➡ .097	TEST 2 ➡ .099	TEST 3 ➡ .099
---------------	---------------	---------------

- RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	0	(0-.04)	5	(.05-.09)	0	(.10-.14)	0	(.15-.19)	0	(OVER .19)	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	------------	---

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

Meets DOH Standards

INSPECTING OFFICER

SIGNATURE 	PRINT NAME Dale L. Cox
TYPE II PERMIT NUMBER/EXPIRATION DATE 280328 11/30/2020	TELEPHONE NUMBER (660) 259-3622

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office
2875 James Boulevard
Poplar Bluff, MO 63901

Col 010328
Lc10

Col. Check

Col 080328
Lc10

Col. Check

Col 080321
Lc10

Col. Check

Col 080327
Lc10

Col. Check

AS IV SERIAL NO: 102455
VERSION NO: 5028

TEST POINT 00659

10/01/20 10:44
Callibration Check
25 10/01/20 10:44 .009

AS IV SERIAL NO: 102455
VERSION NO: 5028

TEST POINT 00659

10/01/20 10:16
Callibration Check
25 10/01/20 10:16 .009

AS IV SERIAL NO: 102455
VERSION NO: 5028

TEST POINT 00659

10/01/20 10:47
Callibration Check
25 10/01/20 10:47 .009

**GUTH LABORATORIES, INC.**

500 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-6470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 20190 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on April 8, 2020, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1206% (w/vol) ethyl alcohol. The expiration date for this lot number is April 6, 2022 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN02271802 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI
DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT
TYPE II
DALE L COX

Is hereby authorized to instruct and supervise operators; train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER, INTOX DMT

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 11/30/2018

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

NUMBER 280328

EXPIRES 11/30/2020

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MO 580-0771 (6-10)

LAB-4 (Re-10)

STATE OF MISSOURI
DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.

Operator COX, DALE
Permit No 280328
Date Issued 11/30/2018 Date Expires 11/30/2020