

RECEIVED

By Tracy Crews at 9:00 am, Aug 04, 2021



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY
ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 087977	NAME OF AGENCY St. Louis County Police Department	DATE OF INSPECTION 07/29/2021
LOCATION OF INSTRUMENT (STREET AND CITY) 5030 GRIFFIN (3RD PCT)		TIME OF INSPECTION 9:09 am

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

- DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)
- TEMPERATURE OF ALCO SENSOR (10°C - 40°C)
- PRINTER WORKING PROPERLY
- TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE
<input checked="" type="checkbox"/> STANDARD SUPPLIER <u>Intoximeters</u>	LOT # <u>AG015503</u> EXP. DATE <u>06/03/2022</u>
<input type="checkbox"/> SIMULATOR TEMPERATURE (34°C ± 0.2°C) _____	SIM. SN _____ SIM. NIST EXP DATE _____

- CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)
 - 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
 - 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
 - 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 <input checked="" type="checkbox"/> .100	TEST 2 <input checked="" type="checkbox"/> .099	TEST 3 <input checked="" type="checkbox"/> .099
---	---	---

- RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS	1	(0-.04)	0	(.05-.09)	0	(.10-.14)	0	(.15-.19)	0	(OVER .19)	0
----------	---	---------	---	-----------	---	-----------	---	-----------	---	------------	---

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

INSPECTING OFFICER	
SIGNATURE 	PRINT NAME PO Neuman, DSN 3072
TYPE ID PERMIT NUMBER EXPIRATION DATE 210074 / 04/06/2023	TELEPHONE NUMBER (636) 529-8210

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office by mail, fax, or email.