



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

MACCESA					
Complete this report in duplicate at the time of Send copy to Department of Health and Senior				nenever instrument is repaired.	
ALCO SENSOR IV SN 111779	PRINTER SN 09B.3589.547	200		TE OF INSPECTION 0/23/2021	
LOCATION OF INSTRUMENT (STREET AND CITY) 110 S Maple Street Galena, Mo 65656			100	E OF INSPECTION 4:56	
CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed val-					
ues where determined.) Unmarked items must be corrected before using instrument.					
DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)					
☑ TEMPERATURE OF ALCO SENSOR (10°C - 40°C)					
PRINTER WORKING PROPERLY					
☑ TIME AND DATE DISPLAYING PROPERLY					
BREATH ALCOHOL ACCURACY STANDARD	os				
☑ SIMULATOR SOLUTION ☐ COMPRESSED ETHANOL-GAS MIXTURE					
STANDARD SUPPLIER Guth	LOT # 20190 EXP. DATE 04/06/2022				
SIMULATOR TEMPERATURE (34°C ± 0.2				OR EXP DATE 12/01/2021	
Elimetricit Elimetricite (crosses) — cimetricite (crosses)					
CALIBRATION CHECK – (ONLY ONE STAR Run three tests using a standard solution. less. Check the box corresponding to the sign of	All three tests must be tandard solution being BETWEEN 0.095% and BETWEEN 0.076% and	e within ±5% of the s used. (PRINTOUT A d 0.105% INCLUSIVE d 0.084% INCLUSIVE	tandard value and TTACHED) E	must have a spread of .005 or	
TEST 1 ☞ .095	TEST 2 ☞ .095		TEST 3 ☞ .095		
✓ RFI DETECTOR OPERATING					
INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)					
REFUSALS 0 (004) 0	(.0509) 0	(.1014)	(.1519) 0	(OVER .19) 0	
List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary). Time Changed					
V					
INSPECTING OFFICER	HOLES PERMIT	TALL TARREST MARKET	产发力和 对性的表现在分词		
SIGNATURE			PRINT NAME Kyle Stults		
TYPE II PERMIT NUMBER/EXPIRATION DATE 290292 12/16/2021			TELEPHONE NUMBER (417) 368-4888		
Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office 2875 James Boulevard					
Poplar Bluf	f, MO 63901				

AS IV Serial no: 111779 Version no: 532B

TEST RECORD 00466

Temp Date Time 210L

Air Blank: 09/23/21 04:56 .000 Calibration Check: 17 09/23/21 04:56 .000

Subject Mame

Monthly Maint
Subject I.II.

Self Test Operator Name: I.D.

K. Stults 290292

Stone Co Jail

AS IV Serial no: 111779 Version no: 5328

TEST RUCORD 00468

Temp Date Time 2101

Air Blank

09/23/21 05:02 ,000

Calibration Check: 18 99/23/21 05:02 .095

Subject Mame

Test 1

Monthly Maint

KStults 290292

Stone Co Jail

AS IV Serial not 111779 Version not 532B

TEST RECORD 00469

Temp Date Time 2181.

7r Blank

69/23/21 **65:63 .60**8

Calibration Check

9 87/20/21 85:03 .895

Monthly Maint

Test 2

KStalts 290292

Stone Co Sail

RS IV Scria) **no: 111779** Version no: **5328**

TEST RULLIRD 00470

Temp Jaio Time 210L

F Manki PW2477 **95:95 .00**9

aliigation Check!

20 69/38/7/ 05:05 .095

Monthly Maint

Tat 3

alo Rames I.D.

KS/n/15290292

Stone Co Jail

nS IV Scriel **no: 111779** Virsion no: **5328**

TEST COORD 00471

ar Pale Time 210L

INTERNAL PERM

Monthly Marnt

RFI

43 I.D.

K. Stults 290292

StoneCotail

monthly Maint
09/23/2021
Stone County Sheriffsoffice
Dapkyle Stults
Permit # 290292
Exp. 12/16/2021



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 20190 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on April 8, 2020, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1206% (w/vol) ethyl alcohol. The expiration date for this lot number is April 6, 2022 at 11:59 PM.

When used in a calibrated Simulator, operating at



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM



PERMIT TYPE II


KYLE G STULTS

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs,

ALCO-SENSOR IV WITH PRINTER

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 (hrough 577.041, RSMo and 306.111 (hrough 306.119 RSMa.

DATE12/16/2019	wante
NUMBER 29.0292	DIRECTOR OF STATE PUBLIC HEALTH LABORATORY
EXPIRES 12/16/2021	Jest U Wille ::
O 500-0771 (G-10)	DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES
	LAO-4 MG-101

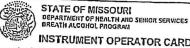

INSTRUMENT OPERATOR CARD

The named cardholder is exhaused to operate an arterial breath alcohol instrument for the determination of the alcoholic content to breath form of oxpired at in Missouri.

Operator STULTS, KYLE Permit No 290292 Dato Issued 12/16/2019 Dato Expiros 12/16/2021

