



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
STATE PUBLIC HEALTH LABORATORY  
BREATH ALCOHOL PROGRAM

**RECEIVED**  
By Tracy Crews at 2:27 pm, Feb 23, 2021

**INTOX DMT MAINTENANCE REPORT**

REPORT #1

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days).  
Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service.  
Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX DMT SN 500288	NAME OF AGENCY MARYVILLE DEPARTMENT OF PUBLIC SAFETY	DATE OF INSPECTION 01/13/2021
LOCATION OF INSTRUMENT (STREET AND CITY) 101 N. VINE		TIME OF INSPECTION 18:21:59

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

**DIAGNOSTIC RECORD**

DATE AND TIME <u>01/13/2021 18:22:01</u>	<input checked="" type="checkbox"/> DETECTOR
<input checked="" type="checkbox"/> PROGRAM	<input checked="" type="checkbox"/> FILTER 1
<input checked="" type="checkbox"/> SAMPLE CHAMBER <u>48.7°C</u>	<input checked="" type="checkbox"/> FILTER 2
<input checked="" type="checkbox"/> BREATH TUBE <u>48.1°C</u>	<input checked="" type="checkbox"/> FILTER 3
<input checked="" type="checkbox"/> PUMP	<input checked="" type="checkbox"/> INTERNAL STANDARD

**BREATH ANALYZER ACCURACY STANDARDS**

SIMULATOR STANDARD  COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER GUTH LOT # 20420 EXP. DATE 09/22/2022

SIMULATOR TEMP (34°C ± 0.2°C) 34.0 SIM. SN SD2281 SIM. NIST EXP DATE 08/31/2021

**CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)**  
Run three tests using a standard. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard being used.

0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE

0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE

0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1: 0.098	TEST 2: 0.098	TEST 3: 0.098
---------------	---------------	---------------

**PERFORM R.F.I. TEST**

**INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:**

REFUSALS: 0	0-.04: 1	.05-.09: 1	.10-.14: 6	.15-.19: 0	OVER .19: 1
-------------	----------	------------	------------	------------	-------------

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY)

INTOX DMT #500288 CONFORMS TO DEPARTMENT OF HEALTH SPECIFICATIONS.

**INSPECTING OFFICER**

SIGNATURE <i>Wayne L Wilson</i>	PRINT FULL NAME WAYNE L WILSON	
TYPE II PERMIT NUMBER 290162	EXPIRATION DATE 07/29/2021	TELEPHONE NUMBER 660-562-3209

RETURN COMPLETED REPORT TO THE Breath Alcohol Program, Missouri Department of Health and Senior Services by mail, fax, or email

**GUTH LABORATORIES, INC.**

590 NORTH 67th STREET • HARRISBURG, PA 17111-4611 • TELEPHONE: 717-564-5470

**CERTIFICATE OF ANALYSIS**

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number **20420** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **September 23, 2020**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.1208%** (w/vol) ethyl alcohol. The expiration date for this lot number is **September 22, 2022** at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of **0.100 g/210L +/- 3%**.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President  
GUTH LABORATORIES, INC.

*NIST Traceability:*

*Testing was conducted using Cerilliant Reference Standard lot number FN03052002 whose values are traceable to NIST.*

*All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.*