

MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

waccus,					
Complete this report in duplicate at the Send copy to Department of Health and	time of the regular monthly p Senior Services; retain origin	preventative maintenal nal in department file.	nce check, and whene	ever instrument is repaired.	
ALCO SENSOR IV SN 030810	PRINTER SN 84.9324.143		DATE OF 07/14	INSPECTION /2022	
LOCATION OF INSTRUMENT (STREET AND CITY 1207 South Main Street El Dorado S) Springs			INSPECTION PM	
CHECKLIST: Place a mark in the box by	each item if found to be sati	sfactory or if operating			
ues where determined.) Unmarked items must be corrected before using instrument.					
DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)					
TEMPERATURE OF ALCO SENSOR (10°C - 40°C)					
PRINTER WORKING PROPERLY					
TIME AND DATE DISPLAYING PROPERLY					
BREATH ALCOHOL ACCURACY STAN	IDARDS				
SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE					
STANDARD SUPPLIER Guth Labor	oratories Inc LC	OT # 21380	EXP. DATE 09/13/2	2023	
SIMULATOR TEMPERATURE (34°C	0 ± 0.2°C)34.00 SIMU	JLATOR SNMP29	946 SIMULATOR E	EXP DATE 02/22/2023	
CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT) Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED) 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE					
TEST 1 • 6 Ø (TEST 2 = 0 (0)	Т	EST3 - OLO	/	
RFI DETECTOR OPERATING					
INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)					
REFUSALS (004)		(.1014)	(.1519)	(OVER .19)	
List any new parts and describe any alte established limits (use other side if neces Replaced 9 volt battery	ration or modification that was sary).	as made to restore th		1.	
NSPECTING OFFICER					
SIGNATURE O	2/1		INT NAME arrod Schiereck		
TYPE II PERMIT NUMBER/EXPIRATION DATE 200225/08/13/2022			LEPHONE NUMBER 417) 876-2313		
2875	h Alcohol Program, MO Depa James Boulevard			heast District Office	

AS IV Serial no: 030810 Version no: 532B AS IV Serial no: 030810 TEST RECORD 00464 TEST RECORD 00465 TEST RECORD 00466 Temp Date Time 210L 9/ Time 210L Temp Date Time 210L 07/14/22 13:29 .000 Subject Test: Auto 07/14/22 13:31 .000 07/14/22 13:33 .000 22 07/14/22 13:29 .000 Calibration Check: 22 07/14/22 13:31 .101 23 07/14/22 13:33 .101 Subject Name Subject I.Ď. Subject I.A. Deerator Name, I.D409 Operator Hame: I.D. 105 Janad Schreecel 1207SMACO 1207 South main 1207 South moin Elmada seuss HDomdo Grange Bonnels Gonne AS IV Serial no: 030810 AS IV Serial no: 030810 TEST RECORD 00468 Temp late Time 210L Time 210L Operator Name, I.D.*/0*9 Ellende Springs (207 South MAIN Et Donnele Spring C

AS IV Serial no: 030810



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 21380 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on September 15, 2021, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1214% (w/vol) ethyl alcohol. The expiration date for this lot number is September 13, 2023 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03052002 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM



PERMIT

JARROD SCHIERECK

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER

fer the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections. 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE		want
9 10 10 10 10 10 10 10 10 10 10 10 10 10	COMMUNICATION CONTRACTOR CONTRACTOR	DIRECTOR OF STATE PUBLIC HEALTH LABORATORY
NUMBER 200225	`\.	Salara de de Oliver à la marie de remotifique.
YPIRES 8/13/2022		for the things
5		DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES
*0771-(6:10).		LAS-4:466-161
		TUTAL SHIP TO



STATE OF MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.

Operator SCHIERECK, JARROD

Permit No 200225

Date Issued 8/13/2020 Date Expires 8/13/2022

