



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 STATE PUBLIC HEALTH LABORATORY  
**ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT**

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN 097445	NAME OF AGENCY MISSOURI STATE PARK RANGERS	DATE OF INSPECTION 09/11/2022
LOCATION OF INSTRUMENT (STREET AND CITY) LAKE OF THE OZARKS STATE PARK KAISER MO		TIME OF INSPECTION 1:51 pm

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)

TEMPERATURE OF ALCO SENSOR (10°C - 40°C)

PRINTER WORKING PROPERLY

TIME AND DATE DISPLAYING PROPERLY

**BREATH ALCOHOL ACCURACY STANDARDS**

SIMULATOR SOLUTION  COMPRESSED ETHANOL-GAS MIXTURE

STANDARD SUPPLIER GUTH LABRATORIES LOT # 21190 EXP. DATE 06/08/2023

SIMULATOR TEMPERATURE (34°C ± 0.2°C) 33.99 SIM. SN MP4945 SIM. NIST EXP DATE 08/16/2023

CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)

Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

- 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
- 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
- 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 ← 0.101

TEST 2 ← 0.101

TEST 3 ← 0.100

RFI DETECTOR OPERATING

**INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)**

REFUSALS	(0-.04)	(.05-.09)	(.10-.14)	(.15-.19)	(OVER .19)
----------	---------	-----------	-----------	-----------	------------

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

**INSPECTING OFFICER**

SIGNATURE

PRINT NAME  
 ELEANORE FERREL

TYPE II PERMIT NUMBER/EXPIRATION DATE  
 210156 08/04/2023

TELEPHONE NUMBER  
 (417) 204-2067

**Return completed report to the:** Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office by mail, fax, or email.

AS IV Serial no: 097445  
Version no: 532B

TEST RECORD 00712

Temp Date Time <sup>g/</sup> 210L

Air Blank:

09/11/22 13:36 .000

Calibration Check:

23 09/11/22 13:36 .101

Subject Name

TEST 1

Subject I.D.

TEST 1

Operator Name, I.D.

FERREL 210156

Location

LOSP KAISER

AS IV Serial no: 097445  
Version no: 532B

TEST RECORD 00713

Temp Date Time <sup>s/</sup> 210L

Air Blank:

09/11/22 13:38 .000

Calibration Check:

23 09/11/22 13:38 .101

Subject Name

TEST2

Subject I.D.

TEST2

Operator Name, I.D.

FERREL 210156

Location

LOSP KAISER

AS IV Serial no: 097445  
Version no: 532B

TEST RECORD 00714

Temp Date Time <sup>s/</sup> 210L

Air Blank:  
09/11/22 13:40 .000  
Calibration Check:  
24 09/11/22 13:40 .100

Subject Name

TEST 3

Subject I.D.

TEST 3

Operator Name, I.D.

FERREL 210156

Location

LOSP KAISER

AS IV Serial no: 097445  
Version no: 532B

TEST RECORD 00715

Temp Date Time 210L  
9/

VOID: RFI  
12 09/11/22 13:41

Subject Name

RAIDO I

Subject I.D.

RAIDO I

Operator Name, I.D.

FERREL ROISY

Location

LOSP KAISER



## GUTH LABORATORIES, INC.

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

### CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 21190 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on June 9, 2021, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1206% (w/vol) ethyl alcohol. The expiration date for this lot number is June 8, 2023 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President  
GUTH LABORATORIES, INC.

***NIST Traceability:***

*Testing was conducted using Cerilliant Reference Standard lot number FN03052002 whose values are traceable to NIST.*

*All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.*



STATE OF MISSOURI  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM



**PERMIT**  
**TYPE II**  
**ELEANORE CARSON**

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

**ALCO-SENSOR IV WITH PRINTER**

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 8/4/2021

*Laura P. Way*

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

NUMBER 210156

*Robt. Kneib*

EXPIRES 8/4/2023

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

 **STATE OF MISSOURI**  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM

**INSTRUMENT OPERATOR CARD**

*The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.*

**Operator** CARSON, ELEANORE  
**Permit No** 210156  
**Date Issued** 8/4/2021 **Date Expires** 8/4/2023

