



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES  
STATE PUBLIC HEALTH LABORATORY  
BREATH ALCOHOL PROGRAM

**INTOX EC/IR II MAINTENANCE REPORT**

REPORT #3

Complete this report at the time of the regular monthly preventive maintenance check (not to exceed 35 days). Complete this report whenever the instrument is serviced or repaired and whenever it is placed into service. Retain the original and send a copy within 15 days to the Breath Alcohol Program, DHSS.

INTOX EC/IR II SN 12681	NAME OF AGENCY SLMPD	DATE OF INSPECTION 06/02/2022
----------------------------	-------------------------	----------------------------------

LOCATION OF INSTRUMENT (STREET AND CITY) 1915 OLIVE ST LOUIS	TIME OF INSPECTION 08:31 CDT
---	---------------------------------

**CHECKLIST:** Place a mark in the box by each item if found to be satisfactory or is operating within established limits. (Write in observed values where determined). Unmarked items must be corrected before using instrument.

<input checked="" type="checkbox"/> <b>DIAGNOSTIC RECORD</b>	
<input checked="" type="checkbox"/> BLANK CHECK	<input checked="" type="checkbox"/> CO2 CHECK
<input checked="" type="checkbox"/> FC 1 TEMP	<input checked="" type="checkbox"/> FLOW CHECK
<input checked="" type="checkbox"/> SRC TEMP	<input checked="" type="checkbox"/> FCB CHECK
<input checked="" type="checkbox"/> DET TEMP	<input checked="" type="checkbox"/> CRC COMP CHECK
<input checked="" type="checkbox"/> BT TEMP	<input checked="" type="checkbox"/> CRC CAL CHECK
<input checked="" type="checkbox"/> STD 2 TEMP	<input checked="" type="checkbox"/> PRINT TEST
<input checked="" type="checkbox"/> ETH CHECK	

**BREATH ANALYZER ACCURACY STANDARDS**

<input type="checkbox"/> SIMULATOR SOLUTION	<input checked="" type="checkbox"/> COMPRESSED ETHANOL-GAS MIXTURE		
<input checked="" type="checkbox"/> STANDARD SUPPLIER	INTOXIMETERS	LOT# AG109802	EXP. DATE 04/08/2023
<input type="checkbox"/> SIMULATOR TEMP (34°C ±0.2°C)	SIM. SN	SIM. NIST EXP DATE	

**CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)**

Run three tests using a standard solution. All three tests must be within +5% of the standard value and must have a spread of .005 or less. Mark the box corresponding to the standard solution being used.

<input type="checkbox"/> 0.10% STANDARD - MUST READ BETWEEN 0.095% AND 0.105% INCLUSIVE
<input checked="" type="checkbox"/> 0.08% STANDARD - MUST READ BETWEEN 0.076% AND 0.084% INCLUSIVE
<input type="checkbox"/> 0.04% STANDARD - MUST READ BETWEEN 0.038% AND 0.042% INCLUSIVE

TEST 1 0.079 g/210L	TEST 2 0.079 g/210L	TEST 3 0.079 g/210L
---------------------	---------------------	---------------------


**INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:**

REFUSALS	0	0-.04	5	.05-.09	0	.10-.14	0	.15-.19	0	OVER .19	0
----------	---	-------	---	---------	---	---------	---	---------	---	----------	---

LIST ANY NEW PARTS AND DESCRIBE ANY ALTERATION OR MODIFICATION THAT WAS MADE TO RESTORE THE INSTRUMENT TO OPERATE SATISFACTORILY AND WITHIN ESTABLISHED LIMITS (USE OTHER SIDE IF NECESSARY).

<b>INSPECTING OFFICER</b>	
SIGNATURE <i>Scott Christian</i>	PRINT FULL NAME CHRISTIAN, SCOTT
TYPE II PERMIT NUMBER 210294	TELEPHONE NUMBER ( 314 ) 444-5345
EXPIRATION DATE 12/10/2023	

**RETURN COMPLETED REPORT TO THE:**  
Breath Alcohol Program, Missouri Department of Health and Senior Services,  
by mail, fax, or e-mail

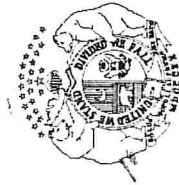

STATE OF MISSOURI  
 DEPARTMENT OF HEALTH AND SENIOR SERVICES  
 BREATH ALCOHOL PROGRAM

2

**PERMIT**  
**TYPE II**

**SCOTT CHRISTIAN**

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

**INTOX EC/IR II**

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 12/10/2021  
 NUMBER 210294  
 EXPIRES 12/10/2023

*Laura P. Day*  
 DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

*Donald A. Kamm*  
 DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES