

MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

THE STATE OF THE S		÷		
Complete this report in duplicate at the time of the Send copy to Department of Health and Senior S	•			whenever instrument is repalred.
ALCO SENSOR IV SN 030810	NAME OF AGENCY El Dorado Springs Police			DATE OF INSPECTION 05/ 16/ 2024
LOCATION OF INSTRUMENT (STREET AND CITY) 1207 South Main Street El Dorado Springs			T	IME OF INSPECTION
CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values				
where determined.) Unmarked items must be corrected before using instrument.				
DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)				
TEMPERATURE OF ALCO SENSOR (10°C - 40°C)				
☑ PRINTER WORKING PROPERLY				
☑ TIME AND DATE DISPLAYING PROPERLY				
BREATH ALCOHOL ACCURACY STANDARDS				
SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE				
STANDARD SUPPLIER Guth Laboratories LOT # 23390 EXP. DATE 10/17/2025				
SIMULATOR TEMPERATURE (34°C \pm 0.2°C) 34.00 SIM. SN MP24946 SIM. NIST EXP DATE 10/17/2025				
less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED) ☑ 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE ☐ 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE ☐ 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE				
TEST 1 Ø TES	ST2 0		TEST 3 🖝 😝	00
☑ RFI DETECTOR OPERATING				
INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)				
REFUSALS (004) (04)	0509)	(.1014)	(.1519)	(OVER .19)
List any new parts and describe any alteration of established limits (use other side if necessary).	r modification that w	as made to restore t	he instrument to	operate satisfactorily and within
INSPECTING OFFICER SIGNATURE		I.	PRINT NAME	
· 0 101	henen	'	Jarrod Schiere	ck
TYPE II PERMIT NUMBER/EXPIRATION DATE 220233/09-12-2024	1	TELEPHONE NUMBER (417) 876-2313		
Return completed report to the: Breath Alcoho by mail, fax, o		artment of Health and	· · · · · · · · · · · · · · · · · · ·	

Ah Glanks 18771672M (1998) 1887 Caple 1 Test 1 Auto-[20] 机探钥索环 上北约3 [40]

orsmator Homes Collin JARROD Schiereck

1207 SOWTH MAIN

Econolo Spengs

Of Other Edition ACMAND Ulas Companie (COST)

1000 2000EN 99844

ulaha Time 210%

90216724 [0195 .999 Coation at tom through

78 (G) (R) (PCA) (B)

Dalajart Maga

JARROD Schrereck

1207 SOUTH MAIN

ELLISANDO SARWAS

内部 (U Serial not 所類) 30 Meny Long and CAMIN

Hate Time 2191

Air Stands 1957/36/2011/97:02 1980 Coldinal has thorby 27 95/16/24 POWY JBI

1207 SOUTH MAIN

EZDONALO Speings

ar it Sorial not 839618 there in mark 18398.

(1987年) | PRODEN | 1 (19646)

Yanga Halis Ying 218K.

335/1(c/24 ESEQUE 999) Calling Wign Clearly C.

THE HE SHY LEFT WAS BROKEN White Lord Space (GROAD)

1891 poyan - 69642

dome Hote Yigo 218%

1207 SOUTH MAIN

Ellando Societos

JARROD Schierack 1207 SOUTH MAIN EZ Domala Sozings



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 23390 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on October 18, 2023, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1207% (w/vol) ethyl alcohol. The expiration date for this lot number is October 17, 2025 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03072301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



580-0771 (6-10)

STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



L'AB-4 (R6-10)

PERMIT TYPE II

JARROD SCHIERECK

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo. DATE 9/12/2022 DIRECTOR OF STATE PUBLIC HEALTH LABORATORY EXPIRES 9/12/2024 DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air to the sound in the soun

Operator SCHIERECK, JARROD

Permit No 220233

Date Issued 9/12/2022 Date Expires 9/12/2024

