

MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

attre.				
Complete this report in duplicate at the time of the regular mo Send copy to Department of Health and Senior Services; retail	onthly preventative maintenance check, and whenever instrument is repaired. In original in department file.			
ALCO SENSOR IV SN 102466 Sn/				
LOCATION OF INSTRUMENT (STREET AND CITY)	TIME OF INSPECTION			
	e satisfactory or if operating within established limits. (Write in observed values			
where determined.) Unmarked items must be corrected before				
DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)				
☐ TEMPERATURE OF ALCO SENSOR (10°C - 40°C) 23°C				
PRINTER WORKING PROPERLY				
TIME AND DATE DISPLAYING PROPERLY				
BREATH ALCOHOL ACCURACY STANDARDS				
SIMULATOR SOLUTION	☐ COMPRESSED ETHANOL-GAS MIXTURE			
STANDARD SUPPLIER Guth Labs				
SIMULATOR TEMPERATURE (34°C ± 0.2°C) 37% SIM. SN 507322 SIM. NIST EXP DATE $01/23/25$				
CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT) Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED) 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE				
TEST 1 = , 098% TEST 2 = ,	097% TEST 3 # , 097%			
☐ RFI DETECTOR OPERATING				
INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)				
REFUSALS (004) (.0509)	(.1014) / (.1519) (OVER.19)			
List any new parts and describe any alteration or modification established limits (use other side if necessary). Lus fru ment is a perating u	that was made to restore the instrument to operate satisfactorily and within different forms.			
Mederal Medical Control of the Contr	LODIN NAME			
GIGNATURE Pressur a There	GARDONY A. West			
TYPE II PERMIT NUMBER/EXPIBATION DATE 230/26 06/14/25	TELEPHONE NUMBER / 573-468-8001			
Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office by mail. fax. or email.				



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 23390 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on October 18, 2023, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1207% (w/vol) ethyl alcohol. The expiration date for this lot number is October 17, 2025 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03072301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.

	## Of the Control of	
	## 10 Serial no: 162466 Usersion no: 532B TEST RECORD 61555 Temp Date Time 216L ### Blank: 11/68/24 13:39 .080 Subject Name Orerator Name, 1.D. Location	
	AS IV Serial no: 102466 Uersion no: 532B TEST RECORD 01554 Switch Blank: 11/08/24 13:35 .000 Subject Name Subject I.D. Drerator Name, I.D.	
•	AS IV Serial no: 182466 Uersion no: 532B TEST RECORD 81553 Temp Date Time 218L Air Blank: Air Blank: 22 11/88/24 13:33 .698 Subject Name Gretator Name, I.D. Location	



MO 580-0771 (6-10)

STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT TYPE II

GREGORY A. WEST

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE6/14/2023	
NUMBER 230126	
EXPIRES 6/14/2025	y <u> </u>

Doula J. Nichelson

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

LAB-4 (R6-10)



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired ali in Missouri.

Operator Permit No WEST, GREGORY 230126

Date Issued 6/14/2023

Date Expires 6/14/2025

