



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired Send copy to Department of Health and Senior Services; retain original in department file.							
NAME OF AGENCY POLICE DEAT DATE OF INSPECTION S. 11. Unas Police Deat 12/11/24							
OCATION OF INSTRUMENT (STREET AND CITY) 106 1 106 20 5 0 5 0 5 0 5 0 0 8 3 0 0 8 3 0 0 8 3 0 0 8 3 0 0 0 8 3 0 0 0 0							
DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)							
TEMPERATURE OF ALCO SENSOR (10°C - 40°C) / 9 °C							
PRINTER WORKING PROPERLY							
TIME AND DATE DISPLAYING PROPERLY							
BREATH ALCOHOL ACCURACY STANDARDS							
SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE							
STANDARD SUPPLIER Guth Labs LOT # 23390 EXP. DATE 10/17/25							
SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34 °C SIM. SN 503322 SIM. NIST EXP DATE $01/23/25$							
CALIBRATION CHECK – (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT) Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 of tests. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED) 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE							
EST 1 098 % TEST 2 099 % TEST 3 099 %							
RFI DETECTOR OPERATING							
NDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: OO NOT INCLUDE SELF-ADMINISTERED TESTS) (004) (.0509) (.1014) (.1519) (OVER .19) (OVER .19) ist any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within stablished limits (use other side if necessary).							
Two trument is operating within D.D.H. specifications.							
SPECTING OFFICER GNATURE GNATURE SURGERY OF WEST PRINT NAME GREY DE C. West TELEPHONE NUMBER 873-468-8001							
eturn completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office by mail, fax, or email.							

AS IV Serial no: 102466 Version no: 532B	ORD 01562	S/ Temp Date Time 218L	UOID: RFI 12 12/11/24 08:43	Subject Name		adbaggt 1. D.	Operator Name, I.D.	\$ C		
AS IV Serial no: 182466 Version no: 532B	TEST RECORD 01561	Temp Date Time 218L	Air Blank: 12/11/24 88:41 .000 Calibration Check:	28 12/11/24 88:41 ,899	Subject Name	Subject I.D.		Operator Name, I.D.	Location	
AS IV Serial no: 102466 Version no: 532B	TEST RECORD 01560	Temp Date Time 210L	Air Blank: 12/11/24 68:38 .000	20 12/11/24 68:38 ,699	Subject Name	Shirt I. I		Operator Name, I.D.	Location	
AS IV Serial no: 102466 Version no: 532B	TEST RECORD 01559	Temr Date Time 218L	Air Blank: 12/11/24 88:35 .888 Calibration Check:	19 12/11/24 88:35 .698	Subject Name	Subject I.D.	3.7	UPerator Name, I.D.	Location	

.



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 23390 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on October 18, 2023, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1207% (w/vol) ethyl alcohol. The expiration date for this lot number is October 17, 2025 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03072301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT TYPE II

GREGORY A. WEST

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE6/14/2023	Mile Massur
	DIRECTOR OF STATE PUBLIC HEALTH LABORATORY
NUMBER 230126	
EXPIRES 6/14/2025	Daven I. Nichelson
MO 580-0771 (6-10)	DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES LAB-4 (R6-10)



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.

Operator W Permit No 23

WEST, GREGORY 230126

Date Issued 6/14/2023 Date Expires 6/14/2025

