

MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in duplicate at the time of Send copy to Department of Health and Senio				nenever instrument is repaired.	
ALCO SENSOR IV SN 111762	PRINTER SN 09B.3589.487			TE OF INSPECTION 9/18/2024	
LOCATION OF INSTRUMENT (STREET AND CITY) Stone County Jail Galena MO				E OF INSPECTION :54 pm	
CHECKLIST: Place a mark in the box by each ues where determined.) Unmarked items must					
DIGITAL READOUT (ALL ELEMENTS OF		ong monument.			
✓ TEMPERATURE OF ALCO SENSOR (10)	°C - 40°C)				
PRINTER WORKING PROPERLY					
☐ TIME AND DATE DISPLAYING PROPER	LY				
BREATH ALCOHOL ACCURACY STANDAR	DS				
SIMULATOR SOLUTION		COMPRESSE	D ETHANOL-GAS	MIXTURE	
STANDARD SUPPLIERGuth	L	OT # <u>23390</u>	EXP. DATE 10	/17/2025	
SIMULATOR TEMPERATURE (34°C ± 0.3	2°C)34.0SIMI	ULATOR SNMP	6027 SIMULATO	OR EXP DATE 10/17/2025	
Run three tests using a standard solution. less. Check the box corresponding to the solution. 0.100% STANDARD - MUST READ E 0.080% STANDARD - MUST READ E 0.040% STANDARD - MUST READ E	standard solution being BETWEEN 0.095% and BETWEEN 0.076% and	used. (PRINTOUT A d 0.105% INCLUSIVE d 0.084% INCLUSIVE	ATTACHED) E E	must have a spread of .005 or	
TEST 1 .100	EST 2 .100		TEST 3 ■ .098		
☑ RFI DETECTOR OPERATING					
INDICATE THE NUMBER OF BREATH TEST (DO NOT INCLUDE SELF-ADMINISTERED T		G RANGES SINCE	THE LAST MAINTE	NANCE REPORT:	
REFUSALS 0 (004) 0	(.0509)	(.1014)	(.1519) 0	(OVER .19) 2	
List any new parts and describe any alteration established limits (use other side if necessary) Monthly Maint. Corrected Time.		vas made to restore	the instrument to o	perate satisfactorily and within	
INSPECTING OFFICER					
SIGNATURE			PRINT NAME Corporal Kyle Stults		
TYPE II PERMIT NUMBER/EXPIRATION DATE 240070 03/22/2026			TELEPHONE NUMBER (147) 357-6116		
	cohol Program, MO Dep es Boulevard off. MO 63901	partment of Health ar	` '	Southeast District Office	

AS IV Serial no: 111762 Version no: 532B

TEST RECORD 00849

9/ Temp Date Time 210L

Air Blank: 09/18/24 15:54 .000 Calibration Check:

22 09/18/24 15:54 .000

Subject Name

Jelf test

Subject I.D.

Mothly Maint
Operator Name, I.D.

K. Stulb 240000

Location

Stone Co Jail

AS IV Serial no: 111762 Version no: 532B

TEST RECORD 00850

Temp Date Time 210L

air Blank:

09/18/24 15:57 .000 Calibration Check: 23 09/18/24 15:57 .100

Subject Name

Test 1

Subject I.D.

Monthly Maint

E. Stults 240070

Location

Stone to Juil

AS IV Serial no: 111762 Version no: 532B

TEST RECORD 00851

Temp Date Time 210L

Air Blank:

09/18/24 16:00 .000 Calibration Check:

23 09/18/24 16:00 :100

Subject Name

Test 2

Subject I.D.

Monthly Maint

K. Stalts 24000

Location

AS IV Serial no: 111762 Version no: 532B

TEST RECORD 00852

9/ Temp Date Time 210L

Air Blank: 09/18/24 16:02 .000

Calibration Check: 24 09/18/24 16:02 .098

Subject Name

Test 3

Monthly Maint

Operator Name: I.D.

K.Stults 240000

Location

Store Lo Bail

AS IV Serial no: 111762 Version no: 532B

TEST RECORD 00853

Temp Date Time 210L

VOID: RFI 12 09/18/24 16:04

Subject Name

ZFI

Subject I.D.

monthly Maint

VStalk 74007

Location

Stone Co Sail

Stone County Sheriff's Office Monthly Maint 09/18/2024 Cpl: K. Stulks

Per mit # 270040 Exp: 03/22/2026



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 23390 of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on October 18, 2023, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1207% (w/vol) ethyl alcohol. The expiration date for this lot number is October 17, 2025 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03072301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT TYPE II

KYLE G. STULTS

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 3/22/2024

DATE	3/22/2024
NUMBER	240070
EXPIRES	3/22/2026

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

Daves I. nichelson

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

LAB-4 (R6-10)

MO 580-0771 (6-10)



The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired a in Missouri.

Operator Permit No

STULTS, KYLE 240070

Date Issued 3/22/2024

Date Expires 3/22/2026

