

MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

74607					
Complete this report in duplicate at the time of the Send copy to Department of Health and Senior S			nce check, and v	whenever instrument is repaired.	
ALCO SENSOR IV SN 030810	NAME OF AGENCY El Dorado Spring	s Police Dept		ATE OF INSPECTION 01/30/2025	
LOCATION OF INSTRUMENT (STREET AND CITY) 1207 South Main Street El Dorado Springs			TI	ME OF INSPECTION	
CHECKLIST: Place a mark in the box by each iter			vithin established	l limits. (Write in observed values	
where determined.) Unmarked items must be corrected before using instrument.					
DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)					
☑ TEMPERATURE OF ALCO SENSOR (10°C - 40°C)					
PRINTER WORKING PROPERLY					
☑ TIME AND DATE DISPLAYING PROPERLY					
BREATH ALCOHOL ACCURACY STANDARDS					
SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE					
STANDARD SUPPLIER Guth Laboratories LOT # 24110 EXP. DATE 03/05/2026					
SIMULATOR TEMPERATURE (34°C ± 0.2°C	c) <u>34.00</u> SIN	n. sn <u>MP24946</u>	SIM. NIS	ST EXP DATE 01/22/2026	
less. Check the box corresponding to the star 0.100% STANDARD - MUST READ BET 0.080% STANDARD - MUST READ BET 0.040% STANDARD - MUST READ BET	TWEEN 0.095% and TWEEN 0.076% and	0.105% INCLUSIVE 0.084% INCLUSIVE	TACHED)		
TEST 1 • 098 TES	T2 - 099	Т	EST 3 🖛 🧯	099	
RFI DETECTOR OPERATING				`	
INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)					
	_ 1	(.1014)	(.1519)	OVER .19)	
List any new parts and describe any alteration or established limits (use other side if necessary). N/A	modification that w	as made to restore th	e instrument to	operate satisfactorily and within	
INSPECTING OFFICER					
BIGNATURE ▶	Phine		aint name Jarrod Schiered	ck	
TYPE II PERMIT NUMBER/EXPIRATION DATE 240209-09/13/2026			TELEPHONE NUMBER (417) 876-2313		
Return completed report to the: Breath Alcoho by mail, fax, o	l Program, MO Depa				

AS IV Serial no: 030810 Version no: 532B

TEST RECORD 00708

Temp Date Time 210L Air Blank: 01/30/25 15:30 .000 Subject Test: Auto 21 01/30/25 15:30 .000

Subject Hame

Black TOST

Subject L.U.

Operator Hame, I.D./09

Jarnad Schiereck

Location

(207 SOUTHMAN)

ELDORADO SAMOS

AS IV Serial no: 030810 Version no: 532B

TEST RECORD 00709

Temp Date Time 210L

Air Blank: 01/30/25 15:32 .000 Calibration Check: 22 01/30/25 15:32 .098

Subject Name

TEST#1

Subject I.D.

Operator Name: I.D./69

Janad Schreeck

Location

(20) SouTH MAN

Albando forings

AS IV Serial **no: 030810** Varsion no: 532B

TEST RECORD 09710

Temp Date Time 210L

Air Blank:

01/30/25 15:33 .000 Calibration Check:

22 01/30/25 15:33 .099

Subject Name

7537#2

Subject I.D.

Operator Mame, I.D. 109

Janacol Schnersch

Location

(20) SOWTH MANN CEDOPARD SNINGS

AS IV Serial no: 090810 Version no: 532%

TEST RECORD 00711

s/ Temp Bate Time 210L

Air Blank: 81/30/23 **15:35 .000**

Calibration Check: 22 01/30/25 15:35 .099

Subject Name

TES7 #3

Subject I.D.

Operator Name, I.D./09

JARROCK Schreeck

Location

(26) South MAIN Ellouado Sprives AS IV Serial no: 030810 Version no: 532B

TEST RECORD 60712

Temp Date Time 2101.

UOID: RFI

12 01/36/25 15:37

Subject Hame

R,F,I, TEST

Operator Nome, I.D./09

JARROCK Schiereck

Location

(20) SOUTH MAN ET DORAGE SPINGS



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 24110 of
Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on March 6, 2024, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.1215% (w/vol) ethyl alcohol. The expiration date for this lot
number is March 5, 2026 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:

Testing was conducted using Cerilliant Reference Standard lot number FN03072301 whose values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT TYPE II JARROD SCHIERECK

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER

for the determination of the alcoholic content of blood from a sal	mple of expired air. Permit issued under the provisions of sections
577.020 through 577.041, RSMo and 306.111 through 306.119	adam / fuli
DATE9/13/2024	DIRECTOR OF STATE PUBLIC HEALTH LABORATORY
NUMBER 240209	Davla I. Nichelson
EXPIRES 9/13/2026	DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES LAB4 (RG-10)

0-0771 (8-10)



The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired all in Missouri.

Operator SCHIERECK, JARROD

Permit No 240209

Date Issued 9/13/2024 Date Expires 9/13/2026

