

RECEIVED

By Tracy Crews at 8:45 am, Jan 30, 2025



MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES
STATE PUBLIC HEALTH LABORATORY

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in duplicate at the time of the regular monthly preventative maintenance check, and whenever instrument is repaired. Send copy to Department of Health and Senior Services; retain original in department file.

ALCO SENSOR IV SN <u>B99362</u>	NAME OF AGENCY <u>509 SFS WAFB</u>	DATE OF INSPECTION <u>28 JAN 2025</u>
LOCATION OF INSTRUMENT (STREET AND CITY) <u>1031 VANDENBERG AVENUE, WHITEMAN AFB</u>		TIME OF INSPECTION <u>0814</u>

CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values where determined.) Unmarked items must be corrected before using instrument.

- DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)
- TEMPERATURE OF ALCO SENSOR (10°C - 40°C)
- PRINTER WORKING PROPERLY
- TIME AND DATE DISPLAYING PROPERLY

BREATH ALCOHOL ACCURACY STANDARDS

- SIMULATOR SOLUTION COMPRESSED ETHANOL-GAS MIXTURE
- STANDARD SUPPLIER GUTH LOT # 22310 EXP. DATE 11 AUG 2024
- SIMULATOR TEMPERATURE (34°C ± 0.2°C) 34°C SIM. SN MP6026 SIM. NIST EXP DATE 2 JAN 2026

CALIBRATION CHECK - (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)
Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)

- 0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE
- 0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE
- 0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE

TEST 1 = <u>.101</u>	TEST 2 = <u>.100</u>	TEST 3 = <u>.100</u>
----------------------	----------------------	----------------------

- RFI DETECTOR OPERATING

INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT: (DO NOT INCLUDE SELF-ADMINISTERED TESTS)

REFUSALS <u>0</u>	(0-.04) <u>0</u>	(.05-.09) <u>0</u>	(.10-.14) <u>0</u>	(.15-.19) <u>0</u>	(OVER .19) <u>0</u>
-------------------	------------------	--------------------	--------------------	--------------------	---------------------

List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).

INSPECTING OFFICER

SIGNATURE <u>Brady L Grimm</u>	PRINT NAME <u>BRADY L GRIMM</u>
TYPE II PERMIT NUMBER EXPIRATION DATE <u>240183</u>	TELEPHONE NUMBER <u>660 687 1459</u>

Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office by mail, fax, or email.

062

02603

Date Time 2101

25 0014 400
26 0128 05 0014 .101

BRADY L GRIMM

240183/29 AUG 2026

BRADY L GRIMM, I.D.

1000000

062

02603

Date Time 2101

25 0014 400
26 0128 05 0014 .101

BRADY L GRIMM

240183/29 AUG 2026

BRADY L GRIMM, I.D.

1000000

062

02603

Date Time 2101

25 0014 400
26 0128 05 0014 .101

BRADY L GRIMM

240183/29 AUG 2026

BRADY L GRIMM, I.D.

1000000

062

02603

Date Time 2101

25 0014 400
26 0128 05 0014 .101

BRADY L GRIMM

240183/29 AUG 2026

BRADY L GRIMM, I.D.

1000000



GUTH LABORATORIES, INC.

590 NORTH 67th STREET • HARRISBURG, PA 17111-4511 • TELEPHONE: 717-564-5470

CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number **22310** of Alcohol Reference Solution for Simulator were analyzed by gas chromatography on **August 16, 2022**, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain **0.1205%** (w/vol) ethyl alcohol. The expiration date for this lot number is **August 11, 2024** at 11:59 PM.

When used in a calibrated Simulator, operating at $34^{\circ}\text{C} \pm .2^{\circ}\text{C}$, this solution will give a breath alcohol analysis instrument reading of **0.100 g/210L** $\pm 3\%$.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President
GUTH LABORATORIES, INC.

NIST Traceability:

*Testing was conducted using Cerilliant Reference Standard lot number **FN03052002** whose values are traceable to NIST.*

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

2

PERMIT
TYPE II
BRADY GRIMM

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

DATE 8/29/2024

NUMBER 240183

EXPIRES 8/29/2026

Mike Massman

DIRECTOR OF STATE PUBLIC HEALTH LABORATORY

Daniel J. Nickelson

DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES

MO 580-0771 (6-10)

LAB-4 (R6-10)

 STATE OF MISSOURI
 DEPARTMENT OF HEALTH AND SENIOR SERVICES
 BREATH ALCOHOL PROGRAM

INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired air in Missouri.

Operator GRIMM, BRADY
 Permit No 240183
 Date Issued 8/29/2024 Date Expires 8/29/2026

