

MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

RECEIVED By Tracy Crews at 10:37 am, Mar 12, 2025

ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

Complete this report in c	duplicate at the time	of the regular monthly	preventative mainten	ance check, and where	ever instrument is repaired.	
Odrid copy to Departmen	nt of Health and Ser	nior Services; retain orig	inal in department file		ever instrument is repaired.	
ALCO SENSOR IV SN 107975		NAME OF AGENCY STODDARD CO	DUNTY SO	DATE OF 03/11	INSPECTION	
LOCATION OF INSTRUMENT (207 SOUTH PRAIRIE	(STREET AND CITY)	MO			INSPECTION	
CHECKLIST: Place a ma where determined.) Unm	ark in the box by eac	h item if found to be sati	sfactory or if operating	within established limits	s. (Write in observed values	
☑ DIGITAL READOUT			g monument.			
✓ TEMPERATURE OF						
PRINTER WORKING		i				
☑ TIME AND DATE DI	SPLAYING PROPE	RLY				
BREATH ALCOHOL AC						
SIMULATOR SOLUT	ΓΙΟΝ		☐ COMPRESSE	D ETHANOL-GAS MIX		
✓ STANDARD SUPPL	IER GUTH LABO	RATORIS	_OT # 24110			
				EXP. DATE 03/25/	2020	
SIMULATOR TEMPE	=RATURE (34°C ± (0.2°C) <u>34.00</u> S	M. SNMP703	SIM. NIST EX	P DATE <u>08/28/2025</u>	
0.100% STAND	ARD - MUST READ ARD - MUST READ	standard solution bein BETWEEN 0.095% an BETWEEN 0.076% an BETWEEN 0.038% an	g used. (PRINTOUT A ld 0.105% INCLUSIVE ld 0.084% INCLUSIVE	TTACHED) : :	t have a spread of .005 or	
TEST 1 .097		TEST 2 🕶 .097		TEST 3 🕶 .096		
RFI DETECTOR OPE	ERATING					
INDICATE THE NUMBER (DO NOT INCLUDE SEL	R OF BREATH TES	TS IN THE FOLLOWIN	IG RANGES SINCE T	HE LAST MAINTENAM	NCE REPORT:	
REFUSALS			1 :			
	(004)	(.0509)	(.1014)	(.1519)	(OVER .19)	
established limits (use oth	ner side if necessar	y).	was made to restore t	ne instrument to opera	te satisfactorily and within	
REPLACED INK RIBB	ON AND CHARG	ED PRINTER.				
					,	
INSPECTING OFFICER						
SIGNATURE SIGNATURE				PRINT NAME		
TYPE II PERMIT NUMBER/EXPIRATION DATE				Zachary Matney		
240194 08/29/2026				(573) 568-4654		
Return completed repor	t to the: Breath Al by mail. f	cohol Program, MO De ax, or email.	partment of Health and	d Senior Services, Sout	heast District Office	

AS IV Serial no: 107975 Version no: 532B TEST RECORD 00753 97 Time 210L Temp Date Air Blank: 03/11/25 14:36 .000 Calibration Check: 27 03/11/25 14:36 .097 Subject Name, Test Subject I.D. Operator Name: I.D. Location / 5050

AS IV Serial no: 107975 Version no: 532B TEST RECORD 00754 Time 210L Temp Date Air Blank: 03/11/25 14:39 .000 Calibration Check: 28 03/11/25 14:39 .097 Subject Name Test 2 Subject I.D. Operator Name, I.D. ZM. Klorney 240194 Location 5050

TEST RECORD 00755

9/
Temp Date Time 210L

Air Blank:
03/11/25 14:41 .000
Calibration Check:
28 03/11/25 14:41 .096

Subject Name

Test 3
Subject I.D.

Operator Name, I.D.

2M. Modray 246191

Lacation: SCSO

AS IV Serial no: 107975

Version no: 532B

AS IV Serial no: 107975 Version no: 532B

TEST RECORD 00756

Temp Date Time 219L VOID: RFI

voina koi -12 **03**/11**/25 14:42**

RFI Test

2 M Matrief 240194

SUSU



CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 24110 of
Alcohol Reference Solution for Simulator were analyzed by
gas chromatography on March 6, 2024, using a Perkin Elmer Gas
Chromatograph Autosystem XL S/N: 610N9030209, and found to contain
0.1215% (w/vol) ethyl alcohol. The expiration date for this lot
number is March 5, 2026 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C .+/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

Ted L. Pauley, President GUTH LABORATORIES, INC.

NIST Traceability:
Testing was, conducted using Cerilliani Reference Standard lot number FN03072301 whose values are traceable to NIST.
All balances are calibrated annually by an outside agency using NIST traceable weights.
Calibration verification is done prior to each use utilizing NIST traceable weights.



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES
BREATH ALCOHOL PROGRAM



PERMIT

ZACHARY MATNEY

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs, and operate the following breath analyzer(s):

ALCO-SENSOR IV WITH PRINTER, INTOX DMT

for the determination of the alcoholic content of blood from a sample of expired air. Permit issued under the provisions of sections 577.020 through 577.041, RSMo and 306.111 through 306.119 RSMo.

_	Through 308.119 RSMo.	and a section
DATE8/29/202		Mile Masson
NUMBER 240194		DIRECTOR OF STATE PUBLIC HEALTH LABORATORY
EXPIRES 8/29/2020		Danla J. Michelson
MO 680-0771 (6-10)	D	IRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES
		LAB-4 Pie-10

