

By Tracy Crews at 8:21 am, Mar 03, 2025



## MISSOURI DEPARTMENT OF HEALTH AND SENIOR SERVICES STATE PUBLIC HEALTH LABORATORY

#### ALCO-SENSOR IV WITH PRINTER MAINTENANCE REPORT

REPORT #7

A STATE OF THE STA				····	
Complete this report in duplicate at the time Send copy to Department of Health and Seni	of the regular monthly pr or Services; retain origina	eventative maintena al in department file.	nce check, and wh	nenever instrument is repaired.	
CO SENSOR IV SN NAME OF AGENCY				TE OF INSPECTION	
111642	Campbell Police	e Departmen		-28-2025	
LOCATION OF INSTRUMENT (STREET AND CITY)			1 110	E OF INSPECTION	
204 Grand Ave Car	netell, MO 639	33		3'07	
CHECKLIST: Place a mark in the box by each item if found to be satisfactory or if operating within established limits. (Write in observed values					
where determined.) Unmarked items must be corrected before using instrument.					
DIGITAL READOUT (ALL ELEMENTS OPERATIONAL)					
TEMPERATURE OF ALCO SENSOR (10°C - 40°C)					
PRINTER WORKING PROPERLY					
TIME AND DATE DISPLAYING PROPERLY					
BREATH ALCOHOL ACCURACY STANDAR	RDS				
SIMULATOR SOLUTION	COMPRESSED ETHANOL-GAS MIXTURE				
STANDARD SUPPLIER GU+h LOT # 24110 EXP. DATE 03-5-2026					
SIMULATOR TEMPERATURE (34°C ± C	0.2°C) 34°C SIN	1. SN <u>SD174</u>	3 SIM. NIS	T EXP DATE <u>02-21-2626</u>	
CALIBRATION CHECK — (ONLY ONE STANDARD IS TO BE USED PER MAINTENANCE REPORT)  Run three tests using a standard solution. All three tests must be within ±5% of the standard value and must have a spread of .005 or less. Check the box corresponding to the standard solution being used. (PRINTOUT ATTACHED)  0.100% STANDARD - MUST READ BETWEEN 0.095% and 0.105% INCLUSIVE  0.080% STANDARD - MUST READ BETWEEN 0.076% and 0.084% INCLUSIVE  0.040% STANDARD - MUST READ BETWEEN 0.038% and 0.042% INCLUSIVE					
TEST 1 • .096	TEST 2 ♥ ; C97		TEST 3 = . O.	17	
RFI DETECTOR OPERATING					
INDICATE THE NUMBER OF BREATH TESTS IN THE FOLLOWING RANGES SINCE THE LAST MAINTENANCE REPORT:					
(DO NOT INCLUDE SELF-ADMINISTERED TESTS)					
REFUSALS (004)	(.0509)	(.1014)	(.1519)	(OVER .19)	
List any new parts and describe any alteration or modification that was made to restore the instrument to operate satisfactorily and within established limits (use other side if necessary).					
INSPECTING OFFICER					
SIGNATURE			PRINT NAME		
· Jerry D. Hargrang			Jerry D Hargraves		
TYPE I PERMIT NUMBER/EXPIRATION DATE  230319 / 12-21-2025			(573) 217-9122		
Return completed report to the: Breath Alcohol Program, MO Department of Health and Senior Services, Southeast District Office by mail, fax, or email.					
Dy man,					

AS IV Serial no: 111642 Version no: 532B

TEST RECORD 00626

Temp Date Time 210L

Air Blank: 02/28/25 23:13 .000 Subject Test: Man 21 02/28/25 23:13 .096

Subject Name Subject I.D.

Jerry Horgraves 230319 Operator Name: I.D.

Campbell Police Dept location

Booking room

AS IV Serial no: 111642 Version no: 532B

TEST RECORD 00627

Time 210L Temp Date

Air Blank: 02/28/25 23:15 .000 Subject Test: Man 21 02/28/25 23:15 .097

Subject Name #7

Subject I.D.

Jerry Hargraws 230319 Operator Name, I.D.

Campbell Police Dept location

Booking room

AS IV Serial no: 111642 Version no: 532B

TEST RECORD 00628

Time 210L Date Temp Air Blank: 02/28/25 23:17 .000

Subject Test: Man 22 02/28/25 23:17 .097

Subject Name

test#3

Subject I.D.

Jeff Horgraves 230319 Operator Name: I.D.

Campbell Police Dept location

Booking room

AS IV Serial no: 111642 Version no: 532B

TEST RECORD 00629

Time 210L Temp Date

VOID: RFI 12 02/28/25 23:18

Subject Name

test#4 RFI Subject I.D.

Jerry Hargranes 230319 Operator Name, I.D.

Campbell Police Dept location

Booking room

Accuracy check AS IV Serial no: 111642 Version no: 532B

TEST RECORD 00625

Temp Date Time 210L

Air Blank:

02/28/25 23:12 .000

Calibration Check: 19 02/28/25 23:12 .097

Subject Name

Accuracy theck is within range Subject I.D.

Jem Hougrand 230319 Operator Name, I.D.

Campbell Police Dept Location

Booking room



### CERTIFICATE OF ANALYSIS

Certified Alcohol Reference Solution for Simulator

Random Samples of Lot Number 24110 Alcohol Reference Solution for Simulator were analyzed by chromatography on March 6, 2024, using a Perkin Elmer Gas Chromatograph Autosystem XL S/N: 610N9030209, and found to contain 0.1215% (w/vol) ethyl alcohol. The expiration date for this lot number is March 5, 2026 at 11:59 PM.

When used in a calibrated Simulator, operating at 34°C +/- .2°C, this solution will give a breath alcohol analysis instrument reading of 0.100 g/210L +/- 3%.

The alcohol and water used in this solution were free of test interfering substances.

> Ted L. Pauley, President GUTH LABORATORIES, INC.

Testing was conducted using Cerilliant Reference Standard lot number FN03072301 whose

values are traceable to NIST.

All balances are calibrated annually by an outside agency using NIST traceable weights. Calibration verification is done prior to each use utilizing NIST traceable weights.



MO 580-0771 (6-10)

## STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM



# JERRY HARGRAVES

is hereby authorized to instruct and supervise operators, train instructors, inspect, calibrate, perform field service and repairs,

is hereby authorized to instruct and supervise operators, train and operate the following breath analyzer(s):  ALCO-SENSOR I	V WITH PRINTER
at blood from a Si	ample of expired air. Permit issued under the provisions of
577.020 through 577.541, 115	Mile Maman
DATE 12/21/2023	DIRECTOR OF STATE PUBLIC HEALTH LABORATORY
NUMBER 230319	David J. Ne Oselso  DIRECTOR OF DEPARTMENT OF HEALTH AND SENIOR SERVICES  LAG 4 (85-10)
EXPIRES 12/21/2025	DIRECTOR OF DEPARTMENT OF THE LAST (R6-10)



STATE OF MISSOURI

DEPARTMENT OF HEALTH AND SENIOR SERVICES BREATH ALCOHOL PROGRAM

## INSTRUMENT OPERATOR CARD

The named cardholder is authorized to operate an evidential breath alcohol instrument for the determination of the alcoholic content in breath form of expired all the block of the content in the second sec in Missouri.

HARGRAVES, JERRY

Permit No 230319

Date Expires 12/21/2025

