

CLOSURE AND USE INTRUCTIONS FOR

BIO-SHIP 602SS FX-11 INFECTIOUS KIT

- I. REMOVE CAP FROM VESSEL AND REMOVE ABSORBENT POUCH.
- II. PLACE PRIMARY CONTAINER/CONTAINERS IN ABSORBENT POUCH CELLS (1 PER CELL).
- III. ROLL UP ABSORBENT POUCH (WITH TUBES IN CELLS) TO PLACE IN VESSEL.
- IV. PLACE CAP ON VESSEL BY PUSHING DOWN SLIGHTLY AND TURNING CLOCKWISE UNTIL CAP HITS STOPS ON VESSEL.
- V. ERECT DIECUT BOX, BOTTOM FIRST BY FOLDING 1 DIECUT FLAP OVER THE OTHER DIECUT FLAP, THEN FOLD OVER THE BOTTOM FLAP, CLOSING WITH THE TABLOCK.
- VI. PLACE VESSEL UPRIGHT IN BOX SO THAT THE BOTTOM OF THE VESSEL SEATS IN THE DIECUT HOLE AT BOTTOM OF THE BOX. PLACE COMPLETED SHIPPING PAPERS IN BOX AND CLOSE BOX BY FOLDING NUMBERED FLAPS 1 (DIECUT FLAP), MAKING SURE THAT THE CAP IS CENTERED IN DIECUT HOLE, THEN FLAP 2, THEN FLAP 3, THEN THE TOP FLAP CLOSING WITH THE TABLOCK.
- VII. PROPERLY MARK AND LABEL THE BOX IN THE SPACES PROVIDED.

NOTICE!

DO NOT PLACE DRY ICE ANYWHERE IN THIS PACKAGE!!

NATIONAL AND INTERNATIONAL REGULATIONS REQUIRE THAT ANY PERSON OFFERING, HANDLING, OR TRANSPORTING DANGEROUS GOODS (INFECTIOUS SUBSTANCES) MUST BE TRAINED.

CHECK APPLICABLE REGULATIONS AND CARRIER REQUIREMENTS FOR PROPER MARKING AND LABELING INSTRUCTIONS--FAILURE TO DO SO CAN RESULT IN FINES AND/OR FRUSTRATED SHIPMENTS.

THE ABOVE PACKAGING CONFIGURATION REPRESENTS THE MAXIMUM VOLUME FOR THIS PACKAGING FOR CERTIFICATION TESTING PUPOSES ONLY. SHIPPER MUST OBSERVE ALL QUANTITY LIMITATIONS AS SET BY APPLICABLE REGULATIONS.

INTERNATIONAL REGULATIONS REQUIRE SUFFICIENT ABSORBENT MATERIAL TO CAPTURE THE COMBINED LIQUID CONTENTS OF THE PRIMARY CONTAINER(S); CONTACT YOUR DISTRIBUTOR FOR ADDITIONAL ABSORBENT/CUSHIONING MATERIAL.