



Missouri Department of Health and Senior Services

P.O. Box 570, Jefferson City, MO 65102-0570 Phone: 573-751-6400 FAX: 573-751-6010
RELAY MISSOURI for Hearing and Speech Impaired and Voice dial: 711

Randall W. Williams, MD, FACOG
Director



Michael L. Parson
Governor

Missouri Department of Health and Senior Services State Public Health Laboratory Mumps Testing

For accurate diagnosis of mumps infection as part of a case investigation being conducted by a Local Public Health Agency or State Department of Health and Senior Services, please collect a Serology and Virus Isolation/RT-PCR specimen as described below. Before collecting and shipping specimens for Mumps testing, please call DHSS at 800/392-0272 (24/7). Note that before any specimen is sent for testing, DHSS staff must first be consulted. After consultation and determination that the patient meets the criteria for testing, you can contact the Missouri State Public Health Laboratory (MSPHL) at 573/751-3334 or 800/392-0272 if you have questions regarding collection and shipping prior to collecting the specimens. This will help ensure that proper specimens are obtained in the right quantity, and that they are packed and transported properly. Mumps kits may be obtained by calling the MSPHL at 573/751-3334.

Serology

Sera should be collected in a red top blood tube as soon as possible after symptom onset for IgM testing. If paired sera are requested, the convalescent sera should be drawn 2-3 weeks later.

Testing of single samples for IgG titer will not be performed. If the suspected cases had received one or more doses of MMR, the IgM response may be missing, delayed, or transient.

Virus Isolation and RT-PCR

Viral mumps specimens should be collected as close to symptom onset as possible, preferably within 1-3 days of onset of parotitis. Particularly among previously immunized suspected cases, mumps virus detection is an important method of confirming the case. The following is the recommended for sample collection.

Oral/buccal swab: Collect fluid by swabbing the buccal cavity (the space between the cheek and teeth). The parotid duct drains in this space near the upper molars.

- a. Massage the parotid gland area just in front of the ear and near the angle of the jaw for 30 seconds prior to collecting secretions on the swab.
- b. Swab the area between the cheek and gum by sweeping the swab near the upper molar to the lower molar area.
- c. Place the swab in viral transport medium and leave the swab in the medium.
- d.

Shipping

Please utilize the MSPHL courier as this provides the fastest means for specimens to arrive at the laboratory. If possible, ship specimens so they will not arrive in Jefferson City on Saturday or a holiday. If the courier cannot be utilized, specimens should be sent by FedEx or United Parcel Service via Priority Overnight transport. **DO NOT SHIP CLINICAL SPECIMENS BY United States Postal Service (USPS).** Ship the kit using a cold pack. If the buccal specimen will not be shipped within 24 hours of collection it should be frozen at -70°C and shipped on dry ice. The serology sample should not be frozen, but shipped on cold packs. Avoid freeze-thaw cycles.

www.health.mo.gov

Healthy Missourians for life.

The Missouri Department of Health and Senior Services will be the leader in promoting, protecting and partnering for health.
AN EQUAL OPPORTUNITY / AFFIRMATIVE ACTION EMPLOYER: Services provided on a nondiscriminatory basis.

Ship To:

Missouri State Public Health Laboratory - Virology

101 North Chestnut Street Jefferson

City, MO 65102

Ph: 573-751-3334

Fax: 573-526-2754