OPTIMIZING PPE with the BATTELLE SYSTEM





- The Battelle Critical Care Decontamination System (CCDS) decontaminates thousands of N95 respirators at once using vapor phase hydrogen peroxide.
- > The Battelle CCDS is designed to work on N95 respirators for the removal of the novel coronavirus (SARS-CoV-2).
- The Battelle CCDS is available to Missouri healthcare providers and first responders at no cost to the state or providers.



EFFICIENT

Tens of thousands of N95 respirators can be decontaminated in one cycle.



FREE TO USE

The CCDS is available

at no cost through an

Emergency Use Authorization
from the Food and Drug

Administration.



AVAILABLE

In partnership with the State of Missouri, FEMA, and HHS, the Battelle system deployed in Jefferson City.



READY FOR REUSE

N95 respirators can be reused up to 20 times without degrading filter performance.



GET STARTED



Visit <u>health.mo.gov/ppe</u> to learn more about how to sign up to use the Battelle system, locate drop off sites, and learn how the process works. Enroll with Battelle at <u>battelle.org/decon</u>.











Sign up with Battelle online

Visit <u>battelle.org/decon</u> to fill out the enrollment form. Battelle will email an agreement, instructions, and a point of contact (POC).



Contact Battelle to get your code

The enrollee completes the agreement and contacts their Battelle POC to receive their 3 digit code for each facility.



Collect and label N95 respirators

N95 respirators must be <u>unsoiled</u> (free of blood, mucus, makeup, etc.) and labeled with a permanent marker.



Bag labeled N95 respirators and ship to the CCDS site

Double bag the labeled N95 respirators and label the shipping box with your 3 digit facility code. Ship via FedEx to 801 Armory Dr, Jefferson City, MO 65109.



Receive decontaminated N95 respirators, ready for reuse

Your shipments are barcoded to ensure chain of custody. You receive your own facility's respirators back decontaminated and ready for reuse.





